General Information
3M™ ESPE® Adper™ Scotchbond™ 1 XT Adhesive is a simple, moist bonding adhesive containing 10% mineral oil.

Adper Scotchbond 1 XT adhesive offers the dental practitioner a wide range of options for bonding procedures. It can be used for direct composite restorations as well as procedures involving porcelains, composite resin repair, self etched enamel, and surface desensitization and bonding of porcelain veneers with 3M™ ESPE® RelyX™ Veneer Cement System and RelyX™ Ceramic Primer.

After curing Adper Scotchbond 1 XT adhesive, it may also be used for amalgam and indirect bonding procedures when combined with 3M™ ESPE® RelyX™ ARC adhesive resin cement. Compatibility with indirect bonding procedures is due to the low film thickness (approximately 10µm) of cured Adper Scotchbond 1 XT.

Adper Scotchbond 1 XT adhesive is available in two delivery systems, a unit dose delivery and a multi-unit viales dispenser.

Use of etchant is critical for both enamel and dentin surfaces.

Indications

• All direct, light-cured bonding procedures.

• Indirect bonding procedures when combined with RelyX ARC Adhesive Resin Cement.

• Root surface desensitization.

• Porcelain and composite repair.

• Bonding veneers in combination with RelyX Veneer Cement

Precautionary Information for Patients and Dental Personnel
3M™ ESPE® Scotchbond™ Universal Etchant contains 34% Phosphoric Acid by weight. Contact with eyes or skin may cause burns. Wear suitable protective clothing, gloves and eye/face protection. In case of contact, flush area with plenty of water and seek medical advice.

Precautionary Information for Patients:
This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylates allergies. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product necessary to stop the reaction and seek medical advice.

Precautionary Information for Dental Personnel:
This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To avoid the risk of allergic reaction, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylics may penetrate commonly used gloves. If product contacts eye, remove and discards glove, wash hands immediately with soap and water and then re-robe. If allergic reaction occurs, seek medical attention as needed.

3M ESPE® Safetydatasheets can be obtained from www.3MESPE.com or contact your local subsidiary.

Hazardous Information

This product is a flammable material.

Recommendations

Use 3M™ ESPE® Vitrebond™ Light Cure Glass ionomer Liner/Base in areas of deep cavity excavation as Class I and II restorations. If pulp exposure has occurred, use a minimum amount of calcium hydroxide followed by application of Vitrebond liner/base. Adper Scotchbond 1 XT adhesive will bond to Vitrebond liner/base.

Adper Scotchbond 1 XT adhesive includes an etch of enamel and dentin as a part of the procedure. It is recommended that the surfaces be left moist after rinsing. Excess surface moisture should be removed by blotting.

Acrylic Scotchbond 1 XT adhesive is recommended for exposure to visible light. The light curing procedures included with this product are similar to the ESPE light curing system. Curing lights should be checked often for proper output using a reliable light metering system.

Air used for drying should be free of oil and water contaminants.

Sensitivity

Some patients may experience transitory postoperative sensitivity. The risk of sensitivity can be minimized by the following measures:

1. Tooth Preparation.

2. Use proper etching. Use of a rubber dam is highly recommended. Use adequate pulp protection. Use a glass ionomer or resin-modified glass ionomer liner/base (3M™ ESPE® Vitrebond) in areas of deep excavation.

2. Adhesive Application

2. Use of compressed air is not recommended to remove pooled water from the preparation after the etch step—blot excess moisture from the preparation using a cotton pellet or mini-sponge.

2. Apply adhesive immediately after blotting.

3. Restorative

3. Place restorative material in increments, curing each increment separately. Adequate cure restorative according to instructions for shade and thickness of restorative and light exposure time.

3. Adjust occlusion carefully. Check for hyperocclusion, particularly in lateral excursion contacts.

Enamel Syringe Assembly

1. Protective eyewear for patients and staff is recommended when using the delivery system.

2. Prepare the delivery system: Remove cap from etchant syringe and SAVE. Twist a blue disposable tip securely onto the syringe. Holding the syringe with the tip away from the patient and any dental staff, express a small amount of etchant onto the dispensing pad or a 2x2 gauze to assure that the delivery system is not clogged.

If clogged, remove the dispensing tip and express a small amount of etchant from the dispensing tip. Replace the dispensing tip and again express debris. If clog remains, discard dispensing tip and replace with a new one. Bend the dispensing tip to a desired angle. Place bend midway along tip. Do not bend dispensing tip at its hub as this may cause the tip to break.

3. Delivery system storage: Remove used dispensing tip and discard. Twist on storage cap. Storage of the delivery syringe with a dispensing tip or without storage cap will allow drying of the etchant and consequent clogging of the system. Replace storage cap with a new dispensing tip at next use.

4. If desired, the etchant may be extruded onto a dispensing pad and applied with a brush or other appropriate instrument.

5. If a liquid etchant is desired, the etchant may be dispensed into a dappen dish and stirred to increase its fluidity.

6. Disinfection: Discard used dispensing tip. Replace syringe cap. See Storage and Use section for disinfection.

Instructions for Use

Dispensing Adper Scotchbond 1 XT Adhesive

Unit Dose:

Attention: To minimize risk of accidental contact with eyes and skin, hold the shaft of the dispensable applicator over the opening where the applicator enters the foil package with your thumb and index finger. Do not activate the unit dose without a dispensable applicator. With your thumb and index finger of the other hand, squeeze the large blaster to transfer the adhesive into the chamber enclosing the applicator. Briefly spin the applicator to fully saturate with adhesive.

Vial:

Pinch the sides of the cap to release the locking mechanism and flip the cap back to reveal the dispensing tip. Squeeze out the exact number of drops you need into the dispensable mixing well. When finished, flip the cap back until it is secured by its locking mechanism.

Directions

Instructions for direct light cure restorations in enamel and dentin

1. Isolation: Rubber dam is the preferred method of isolation.

2. Cavity preparation: Prepare cavity with minimal tooth reduction. Bevel cavosurface enamel margins.

3. Etching: Apply etchant to enamel and dentin. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini-sponge. The surface should appear glistening without pooling of water.

4. Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents. Light cure for 10 seconds.

5. Restorative placement, cure and finishing: Refer to manufacturer's instructions for placement, cure and finishing of restorative materials.

Instructions for bonding porcelain veneers

1. Isolation: Adhesive should have been etched using hydrofluoric acid with the dental etching system. Apply RelyX Ceramic Primer (No. 2721) to the bonding surface of the veneer. Dry for 5 seconds.

2. Clean the prepared teeth in preparation for sealing and bonding using a plain flour of pumice slurry. Rinse thoroughly and dry.

3. Try-in veneer with RelyX™ Try-In Paste, manufactured by 3M ESPE. After try-in, isolate from adjacent teeth with clear matrix strip.

4. Etching: Apply etchant to both enamel and dentin. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini-sponge. The surface should appear glistening without pooling of water. An additional 15 second etch may be required for teeth that were not prepared using a diamond or bur. Residual organic matter can also require additional etch time.

5. Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents. Light cure for 10 seconds.

6. Adhesive application to veneer: Apply 1 coat of adhesive to the acid etched, silane treated veneer. Dry thoroughly. Do not light cure.

7. Luting material application to veneer: Apply RelyX Veneer Cement to the bonding surface of veneer.

8. Sealing and curing: Carefully seal the veneer. Clean excess luting cement from the veneer margins. Cure each area of the veneer for times recommended by the luting cement manufacturer. We recommend curing the gingival margin first, followed by the body and the incisal margin. Avoid direct contact with the light-guide during initial curing.

Instructions for bonding composite and set amalgam

1. Isolation: Rubber dam is the preferred method of isolation.

2. Roughen the existing material: Roughen the surface of existing amalgam or composite using either a bur, diamond or a sandblast technique.

3. Etching: Apply etchant to enamel, dentin and existing restorative material. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini sponge. The surface should appear glistening without pooling of water. An additional 15 second etch may be required for teeth that were not prepared using a diamond or bur. Residual organic matter can also require additional etch time.

4. Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents. Light cure for 10 seconds.

5. Restorative placement: Refer to manufacturer's instructions for placement, cure and finishing of restorative materials.

Instructions for porcelain repair

1. Isolation: Rubber dam is the preferred method of isolation.

2. Roughen the existing material: Roughen the surface of existing amalgam or composite using either a bur, diamond or a sandblast technique.

3. Etching: Apply etchant to enamel, dentin and existing restorative material. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini sponge. The surface should appear glistening without pooling of water. An additional 15 second etch may be required for teeth that were not prepared using a diamond or bur. Residual organic matter can also require additional etch time.

4. Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents. Light cure for 10 seconds.

5. Restorative placement: Refer to manufacturer's instructions for placement, cure and finishing of restorative materials.

Instructions for bonding veneer

1. Isolation: Rubber dam is the preferred method of isolation.

2. Clean the prepared teeth in preparation for sealing and bonding using a plain flour of pumice slurry. Rinse thoroughly and dry.

3. Try-in veneer with RelyX™ Try-In Paste, manufactured by 3M ESPE. After try-in, isolate from adjacent teeth with clear matrix strip.

4. Etching: Apply etchant to both enamel and dentin. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini-sponge. The surface should appear glistening without pooling of water. An additional 15 second etch may be required for teeth that were not prepared using a diamond or bur. Residual organic matter can also require additional etch time.

5. Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents. Light cure for 10 seconds.

6. Adhesive application to veneer: Apply 1 coat of adhesive to the acid etched, silane treated veneer. Dry thoroughly. Do not light cure.

7. Luting material application to veneer: Apply RelyX Veneer Cement to the bonding surface of veneer.

8. Sealing and curing: Carefully seal the veneer. Clean excess luting cement from the veneer margins. Cure each area of the veneer for times recommended by the luting cement manufacturer. We recommend curing the gingival margin first, followed by the body and the incisal margin. Avoid direct contact with the light-guide during initial curing.

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