## JET BLUE BITE

### Definition

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## JET BLUE BITE

### Definition

JET BLUE BITE is an A-silicone-based impression material for use in dentistry with an automatic mixing device.

### Material type

Polyvinylsiloxane, addition-type.

JET BLUE BITE *fast* / *superfast*


Base: blue / Catalyst: light-blue

### Indications


Impression material for bite registration and / or registration of centric occlusion.

### Clinical time

#### JET BLUE BITE *fast*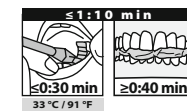

#### JET BLUE BITE *superfast*

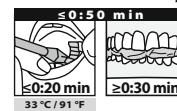
### Contra-indications

Provided that the product is used as directed, there are no known contra-indications.

### Side effects / Interactions

Polyvinylsiloxanes are biologically compatible with no known harmful reactions or secondary effects on patients and/or dental personnel. Impression materials are intended to set in the mouth of the patient, however, they should not remain in the mouth for more than twice the recommended setting time. Although they have high tear strength, care should be taken that no portion of the material remains in the interdental spaces or the sulcus.

### Mixing tip / Spreader tip

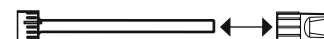Disinfect after use by wiping the Mixing tip with a disinfectant (acidic glutaraldehyde). Store in horizontal position. Always store the cartridge with the Mixing tip attached. This seals the cartridge until it is used again and prevents contamination.

Only remove mixing tip directly before the device is to be used again, check that material is ejected evenly and attach new mixing tip.

### Bite registration

Dispense the bite registration material directly onto the occlusal surface of the lower teeth and move mandible into central occlusion. Always check intraorally if the material has set before removing the registration. High temperatures accelerate and low temperatures slow down setting. Inspect the models for projections (bubbles) from voids in the impression and remove these bubbles before articulating the models since these projections may prevent the proper seating of the bite registration. Once the impression of the bite has set and has been removed from the mouth, it is vital that all gingival contact be trimmed down with a sharp knife or with a trimmer to remove any undercuts. Trimming the bite registration in this manner will prevent any rebound when the models are articulated. Removing the gingival contact will also permit visual inspection to make certain that the bite registration is properly seated.

The material may be used in the „closed mouth“ or „pre-occluded“ technique. Guide the patient into centric **before** applying JET BLUE BITE. Once the proper intercuspation is confirmed, remove the spreader tip from the end of the mixing tip and inject JET BLUE BITE into the interocclusal space over the preparation to create an occluso/buccal index.

## JET BLUE BITE

**Important:** Skin secretion, latex gloves and surfaces contaminated by latex gloves may interfere with the setting process of polyvinylsiloxanes. Only touch material and the surfaces of which the impression is taken (teeth, preparations, retraction cords) with thoroughly washed, rinsed and dried hands (wash for 15 s with a detergent, rinse with water for another 15 s and dry) or use vinyl gloves. Compounds containing eugenol or hemostatic substances can also impede perfect setting. If hydrogen peroxide is used for disinfection, it is recommended to thoroughly rinse with lukewarm water afterwards in order to avoid bubble formation.

### Disinfection

The impression should be rinsed under (cold) running tap water after removal from the mouth. After rinsing, disinfection with a suitable commercial dental disinfectant solution (acidic glutaraldehyde) will not affect the impression surface or dimensions. Acrylic trays should be protected against water absorption.

### Shelf life and storage

JET BLUE BITE fulfils the intended purpose at least until the expiry date, which is displayed on the containers. Storage: well sealed at temperatures 15–23 °C / 59–73 °F and 50 % relative air humidity. Bite registrations should be stored at normal room temperature, avoid exposure to heat and sun.

### Marking

The expiry date and **LOT** number are shown on the package and cartridge.

### Technical data ISO 4823:2000

Measurements are based on 23 °C / 73 °F room temperature and 50 % relative humidity.

#### JET BLUE BITE fast

Mixing time (15 ml): 0:15 min  
Total working time: 0:50 min

#### JET BLUE BITE superfast

Mixing time (15 ml): 0:15 min  
Total working time: 0:45 min

### Instructions for use date

03-2013

Only supplied to dentists and dental laboratories or upon their instructions.

### Caution

Federal law restricts this device to sale by or on the order of a dentist.

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JET BLUE BITE is a trademark of Coltène/Whaledent
JET BLUE is a registered trademark of Coltène/Whaledent AG, internationally, excluding the U.S.
### Coltène/Whaledent AG