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
Honigum-MixStar Putty is an innovative, addition curing silicone-based precision impression material. Honigum-MixStar Putty is available in the readily filled MixStar cartridge to be used in DMG MixStar and other commercially available mixing devices.

Indication


Crown and bridge impression, inlay and only impressions

Handling of the MixStar-Cartridge


Use in DMG MixStar-eMotion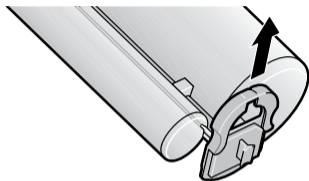


1. Slide locking mechanism into open position. Do not remove locking mechanism from the cartridge.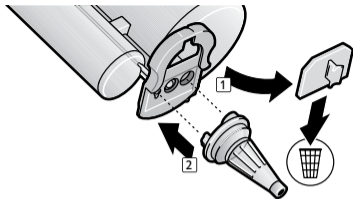

2. Remove and discard the cap on the openings of base and catalyst outlets and attach a new mixing tip.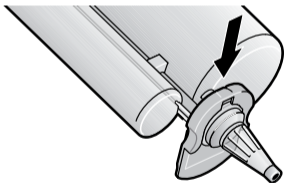

3. Secure the mixing tip by sliding the locking mechanism into locked position.

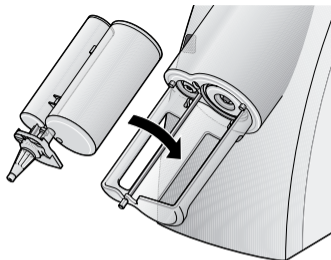4. Insert the cartridge into the MixStar machine. A brief acoustical signal confirms the correct insertion of the cartridge.

- * Please be sure to follow the manufacturer's directions for the respective mixing device you are using.
- * Fresh cartridges are ready for use after the first 3 cm of impression material have been extruded and discarded.

- * After the application, leave the used tip on the cartridge to cap unused material.

Use in Other Mixing Units

If a MixStar cartridge is used with third party mixing devices insert the cartridge into the device first without the mixing tip. Otherwise appropriate mixing of the material cannot be guaranteed. In this case proceed as described in the section »Changing the mixing tip«, and follow the operating instructions for the device used.

Changing the mixing tip

Attach a new mixing tip before each mixing process. Remove the old mixing tip and any hardened material blocking the cartridge opening. First place the mixing tip on the hexagonal mixer shaft and secure it with a light turn, so that the shaft engages in the mixing tip. Then firmly attach the mixing.

Recommended Use

Impression Tray

Honigum-Putty can be used with all commercially available trays. In order to improve the material's adhesion to metal or plastic trays a tray adhesive for VPS silicones can be used.

Impression Taking

Load the impression tray with material by tilting the tray slightly under the mixing tip. In order to avoid air bubbles leave the mixing tip immersed in the impression material during the application. Introduce the tray into the mouth no later than 1.45 minutes after you started the mix, keep that position for at least 3.15 minutes, and then remove the tray from the mouth. Do not combine Honigum with condensation curing silicone or polyether impression materials.

Special Techniques

Double Mixing Technique

Recommended Material Combination:

Honigum-MixStar Putty + Honigum-Light

Load the tray with Honigum-MixStar Putty. While the tray is being loaded by the assistant the dentist can begin to syringe the preparation with Honigum Light. Loading of the tray and syringing should be completed at the same time as the higher temperature in the mouth leads to a quicker set of the wash impression material as compared to the tray material. Position the loaded tray in the mouth and take the impression. After the material has set remove the impression from the mouth.

Correction Impression

Recommended Material Combination:

Honigum-MixStar Putty + Honigum-Light or Honigum-Light Fast

Cut generous pressure relief channels for the wash impression and remove imperfections. After a try-in, clean the pre-impression carefully with water and dry. Apply the correction impression material around the preparation and into the pre-impression. Introduce the tray into the mouth and briefly exert slight pressure. After the material has set remove the tray. To make sure that both materials bond properly no more than 30 minutes should pass between the preliminary and the correction impression. As an alternative the double mixing impression technique can be applied.

Removal

Remove the impression parallel to the longitudinal axis of the prepared teeth. After removal, thoroughly clean with water and disinfect the impression.

Disinfection

Use only dipping solutions for the disinfecting of the impression. We recommended a 2% glutaraldehyde solution. Please ensure that the used disinfectant is indicated for use with addition curing silicone impression materials.

After removal from the mouth, clean the impression under running water for 15 seconds. Leave the impression in the disinfectant for about 10 to 15 minutes. Then rinse off the disinfectant for 15 seconds under running water.

For further information refer to the comments issued by the DGZMK. Comments of the DGZMK 3/93 V 1.0; www.dgzmk.de

ADA recommends to follow Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; www.ada.org

Casting and Galvanizing

After disinfection, store in a dry location at normal room temperature (max. 25 °C/77 °F). A cast can be made at the earliest 30 minutes after

the impression was removed from the mouth. We recommend using a special purpose hard plaster (ISO 6873, Type 3; e.g. Moldano, Bayer) for the cast or an ultra hard dental plaster for the forms (ISO 6873, Type 4; e.g. Fujirock II, GC).

The impressions can be copper- or silver-plated.

Storage of the Impression

Store dry impression at normal room temperature (max. 25 °C/77 °F). The impressions are stable and can be poured even after months without loss of accuracy.

Please Note

- * Never allow the impression to come in contact with organic solvents or liquids containing same as this may result in swelling of the material and therefore may lead to inaccuracies.
- * Possible residues of retraction aids can increase the setting time.
- * Do not touch the impression material with latex gloves. The use of PVC or PE gloves is recommended.

Specifications

Working time*	1:45 min
Recommended time in mouth*	3:15 min
Dimensional change after 24 h	≤ 0.5 %
Compression set	≤ 0.6 %
Strain in compression	~ 2.0 %

* Working time and time in mouth specifications apply to material preparation in a DMG MixStar. The mixing of the material in other devices may require different times. The indicated working time reflects normal room temperature (23 °C/73 °F) and relative humidity (50 %) conditions. Higher temperatures shorten and lower temperatures increase this time value.

Composition

Vinyl siloxanes, silicon dioxide, food pigments, additives, platinum catalyst

Storage

Store and use in dry location at normal room temperature not exceeding 25 °C/77 °F. Storage of the material in the refrigerator or freezer is NOT recommended. Leave used tip on the cartridge after use to cap unused material. Do not use after expiration date.

Classification

Honigum-Putty: ISO 4823:2000, Type 0: Putty consistency

Packaging

1 cartridge @ 380 ml paste, 10 mixing tips
REF 919374
USA, please use REF 999374

5 cartridges @ 380 ml paste, 50 mixing tips*
REF 919375

1 bottle @ 10 ml Tray-Adhesive REF 909394
USA, please use REF 999394

** Not available in the US.*

Federal law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

Keep away from children! For dental use only!

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