Dynamix® speed

Artikel-Nr.: / Item no.: / Réf : / Artículo n.º: / Codice: /
 Ref.º: / Artikelnr.: / Artikelnummer: / Delnr.: /
 Artikkelnr.: / Tuotenumero: / Αρ. Προϊόντος:
66045779

CE

- (DE)** Gebrauchsanweisung
- (GB)** Instructions for use
- (FR)** Mode d'emploi
- (ES)** Instrucciones de uso
- (IT)** Istruzioni per l'uso
- (PT)** Instruções de uso

- (NL)** Gebruiksaanwijzing
- (SE)** Bruksanvisning
- (DK)** Brugervejledning
- (NO)** Bruksanvisning
- (FI)** Käyttöohjeet
- (GR)** Οδηγίες χρήσης

HKG-Layout 99000750/02
 GBA_Dynamix speed

Heraeus

Dateiname: 99000750_02_W06711_GBA Dynamix speed_V8_X3				
SAP-Nr: 99000750	Version 02	HK-Toolbox-Nr: W06711	Format: 420 x 297 mm	Falzmaß: 210 x 297 mm
Projektmanager: Michael Helken		HKG-Version: V8, 7.AK	Datum 26.10.2011	HKG Freigabe am 26.10.2011
EURO / Pantone C M Y K		Sonderfarben	Druckverfahren: Digitaldruck varnish DieCut var.Data	

- DE
- GB
- FR
- ES
- IT
- PT
- NL
- SE
- DK
- NO
- FI
- GR

Dynamix® speed

Artikel-Nr.: 66045779

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Inhalt

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- 1 -

Contents

1.	Introduction	2
1.1	Symbols	2
1.2	EC declaration of conformity	2
2.	Safety	3
2.1	Proper Use	3
2.2	Ambient conditions for safe operation	3
2.3	Ambient conditions for storage and transport	3
2.4	Hazard and Warning Information	3
2.5	Authorized Individuals	3
2.6	Liability Exclusion	4
3.	Product Description	4
3.1	General Description	4
3.2	Assembly and Controls	4
3.3	Standard Delivery	5
3.4	Accessories	5
4.	Commissioning	5
4.1	Setup	5
4.1.1	Bench Top Unit	5
4.1.2	Wall Mounting	5
4.1.3	Setup Recommendations	5
4.2	Connection and Starting	5
4.3	Remove Shipping Restraint	5
5.	Operation	6
5.1	Cartridge Installation and Replacement	6
5.1.1	Select mixing program	6
5.2	Extrusion	7
5.3	Fill Level Indicator and Remaining Amount	7
5.3.1	Fill Level Indicator	7
5.3.2	Remaining Amount Detection	7
6.	Cleaning / Maintenance	7
6.1	Cleaning the Interior	7
6.2	Disinfection	7
6.3	Plungers' Disc Replacement	8
6.4	Unit Cover Replacement	8
6.5	Fuse Replacement	8
6.6	Spare Parts	8
7.	Scope of Supply	8
8.	Delivery Form	8
9.	Troubleshooting	9
10.	Technical Specifications	10
11.	Warranty	10
12.	Disposal Information	10
12.1	Disposing of Consumables	10
12.2	Instrument Disposal	10
12.2.1	Disposal Information for EU Nations	10

1. Introduction

1.1 Symbols

The following symbols are employed in these instructions and on the unit itself:

On the unit:
Adhere to the instructions for use!

In these instructions:
“Danger”
Serious risk of injury!

Electrical current
This indicates a hazard due to electrical current.

Attention
Failure to observe the associated information can result in damage to the unit.

Note
This provides the operator with useful information to make working with the unit easier.

Please observe the operating time.
After 2 minutes of operation, you must pause for at least 5 minutes.

Please observe a pause of 2 hours after the extrusion of 3 complete cartridges (approx. 30 impressions with a medium-size tray) in immediate succession. During the pause the device can remain switched on.

The unit complies with current EU directives.

Within the EU, this device is subject to the provisions of the WEEE directive.

- Required actions, correct order
- Reaction
- Listing

Other symbols are explained as they occur.

1.2 EC declaration of conformity

We, Heraeus Kulzer GmbH, hereby declare that the present product:

Dynamix speed

Item nos.: 66045779

complies with the following European directives:

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2. Safety

Using these operating instructions as a starting point, instruct all operators of the unit with regards to the area of application, the possible hazards during operation, and the proper operation of the unit.

Please have these operating instructions readily available for the operators.

2.1 Proper Use

The Dynamix *speed* is designed solely for the extrusion of 2-component dental impression materials.

The Dynamix *speed* works with 5 : 1 cartridges or tubular bags in reinforcing cartridges from various manufacturers.

Dynamic mixing nozzles with hexagonal head drives are used for mixing.

Only the mixing nozzles and other system parts (e.g. fixation ring, supporting cartridges etc.) specified by the impression material manufacturer may be used.

The use of other mixing nozzles and system parts may cause malfunction.

Always comply with all safety and processing instructions provided by the manufacturer!

Only the mixing nozzles specified by the manufacturer may be used.

2.2 Ambient conditions for safe operation

Safe operation of the unit can only be ensured under the following ambient conditions:

- Indoors;
- Up to an altitude of 2,000 m above sea level;
- At an ambient temperature range between 5 – 40°C [41 – 104°F] *);
for reaching optimum product properties of the impression materials the temperature requirements are valid from those manufacturers. Please also see chapter 4.1.3. Setup Recommendations;
- At a maximum relative humidity of 80% at 31°C [88°F], dropping in a linear manner to 50% relative humidity at 40°C [104°F] *);
- With mains power where the voltage fluctuations do not exceed 10% of the nominal value;
- Under contamination level 2 conditions;
- Under over-voltage category II conditions.

*) Between 5–40°C [41–104°F] the unit can be operated at a relative humidity of up to 80%.

At temperatures between 31 – 40°C [88 – 104°F], the humidity must decrease proportionally in order to ensure operational readiness (e.g., at 35°C [95°F] = 65% humidity; at 40°C [104°F] = 50% humidity).

The unit may not be operated at temperatures above 40°C [104°F].

2.3 Ambient conditions for storage and transport

Ensure the following ambient conditions during storage and transport:

- Ambient temperature -20 – +60°C [-4 – +140°F],
- Maximum relative humidity 80%.

2.4 Hazard and Warning Information

Only intended for indoor use. The unit is only designed for dry applications and may not be operated or stored outdoors or under wet conditions.

The unit may only be operated with a power cord equipped with a plug appropriate to the local power supply.

Regularly inspect connecting lines and hoses (e.g., the power cord) for damage (e.g., kinks, cracks, porosity) or signs of aging.

Units exhibiting damaged connecting lines, hoses, or other defects must be taken out of service immediately.

Portable and mobile communication devices (e.g. cell phones) can affect the functioning of the unit.

The unit must not be placed directly next to other pieces of electrical equipment.

If this is unavoidable, the unit and the other items of electrical equipment should be checked to rule out any malfunctioning caused by electromagnetic interference.

Always unplug the unit from the wall outlet before beginning any work on the unit's electrical components.

Note any hazard or warning information provided by the material manufacturer.

Do not operate in an explosion-prone area.

2.5 Authorized Individuals

This product may not be used by minors under the age of 14.

Only properly trained individuals may operate and service the unit. Any repairs not specifically described in these operating instructions may only be performed by a qualified electrician or authorized dealers.

2.6 Liability Exclusion

Heraeus Kulzer GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is used for any purposes other than those cited in the instructions for use;
- ▶ The product is altered in any way other than those alterations described in the instructions for use;
- ▶ The product is repaired by other than an authorized facility or if any but Heraeus OEM parts are used;
- ▶ The product continues to be used, despite obvious safety faults or damage;
- ▶ The product is subjected to mechanical impacts or is dropped.
- ▶ We do not accept any liability for damage caused by mixing tips, foil bags or cartridges splitting.

3. Product Description

3.1 General Description

The Dynamix *speed* unit for mixing impression materials facilitates extrusion and mixing of 2-component dental impression materials.

5 : 1 cartridges or foil bags in cartridge holders from different manufacturers can be used in the unit.

With Dynamix *speed*, impression materials marked with the symbols  or  can be mixed and extruded in half the time of conventional impression materials using the corresponding mixing program.

The mixing programs control the feed rate of the plungers and the rotational speed of the hexagonal head.

Only use mixing nozzles which are designated for the impression material by the manufacturer.

Cartridges that are not marked with these symbols must be processed using the "ST" program.

A "remaining amount detector" automatically retracts the plungers' disc to the start position so that the cartridges can be exchanged quickly.

3.2 Assembly and Controls

- (1) Fill level indicator
- (2) "Impression tray" advance key (normal extrusion speed)
- (3) "Impression syringe" advance key (slow extrusion speed)
- (4) Retract key
- (5) Yellow LED, residual level indicator
- (6) Green LED, unit ON
- (7) Red LED, unit cover open
- (8) Unit cover
- (9) Plungers' disc
- (10) Spindle covers
- (11) Spindles
- (12) Catch

- (13) Hexagonal end
- (14) Cartridge bed
- (15) Control keys and LED display for mixing programs
- (16) Mains switch
- (17) Fuse
- (18) Mains connection
- (19) Power cord
- (20) Cartridge / Cartridge with tubular bag
- (21) Mixing nozzle
- (22) Service connector

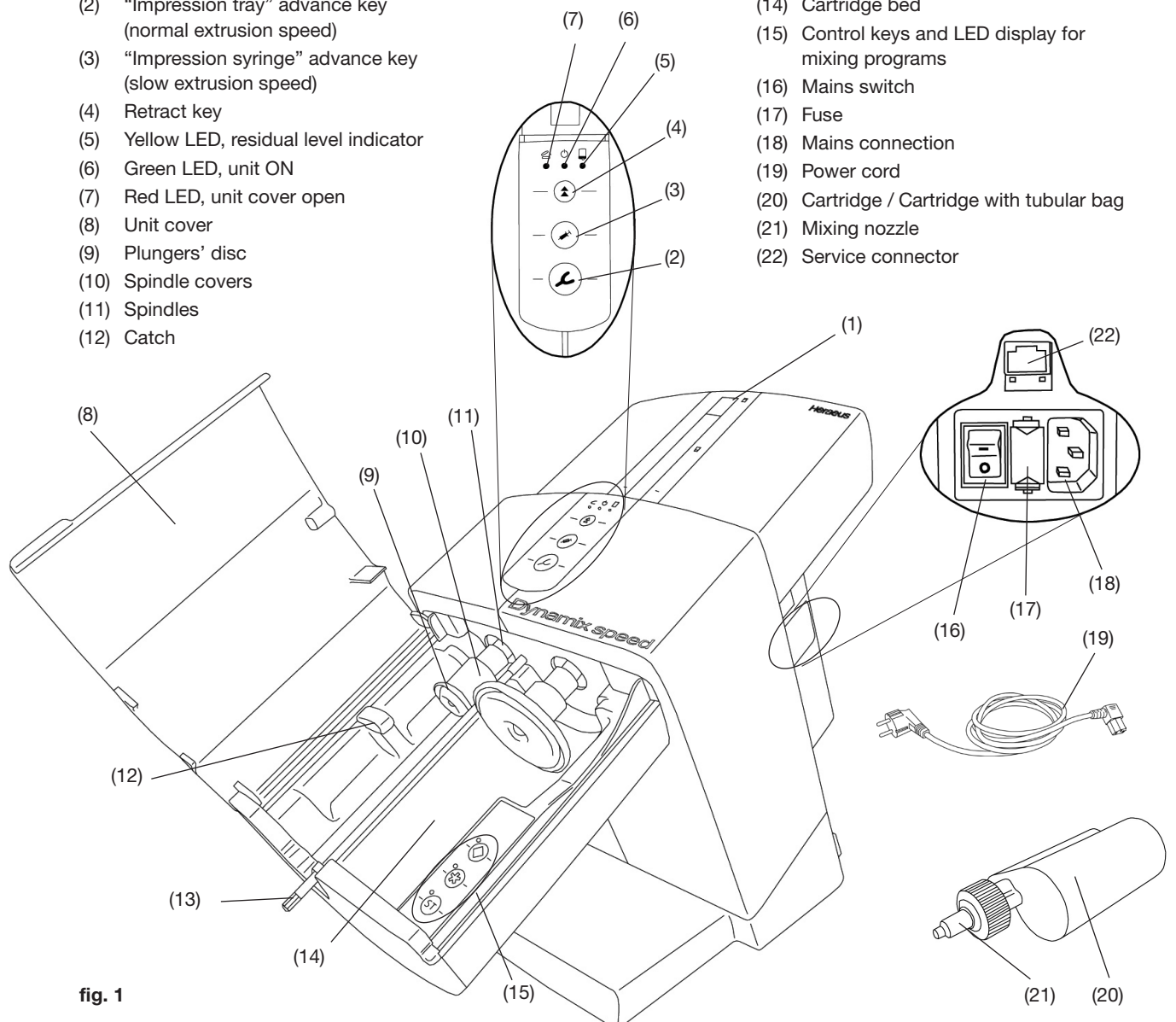
fig. 1

3.3 Standard Delivery

- 1 Dynamix *speed* mixing unit
- 1 Power cord
- 1 Instructions for use

3.4. Accessories

Wall mounting kit

4. Commissioning

4.1 Setup

The Dynamix *speed* can be used as a bench top unit, although it is also suitable for wall mounting.

4.1.1 Bench Top Unit

Select a stable underlay on which to set the unit up.

4.1.2 Wall Mounting

With the aid of the wall mounting kit (refer to the accessories), the unit can be securely mounted on a wall. The mounting instructions are included in the mounting kit.

4.1.3 Setup Recommendations

Operate the unit at room temperature 21 – 25°C [70 – 77°F].

Depending on the information from the impression material manufacturer, the impression material is to be prepared prior to usage in a room temperature between 21 – 25°C [70 – 77°F] for reaching optimum product properties.

Because of this, it is recommended to use Dynamix *speed* at the same temperature. Impression material that is too cold, especially with temperatures less than 21°C [70°F], can result in Dynamix *speed* changing automatically to a lower supporting speed because of safety reasons. Please also see chapter 9: Troubleshooting.

When setting up the unit, please note the following:

- Do not place the unit under a heat source.
- Do not place the unit in front of open windows.
- Do not set the unit up where it will be subjected to direct sunlight.

4.2 Connection and Starting

Before connecting the unit to the wall outlet, make sure the voltage information on the nameplate corresponds to your local power supply.

- Connect the provided power cord (19, fig. 1) to the main connection socket (18).
- Plug the power cord into the wall outlet.
- Switch the unit on at the main power switch (16).

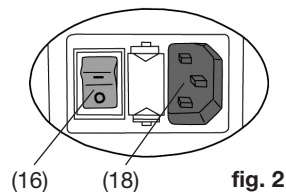
fig. 2

4.3 Remove Shipping Restraint

The shipping restraint must be removed before the unit may be operated:

- The unit has been connected and switched on as described in Section 4.2.
- Close the unit cover.
- Press the retract key (4).
- The plungers (9, fig. 1) retract to their home position where they stop.
- Open the unit cover (8, fig. 1).
- Remove the shipping restraint (23).

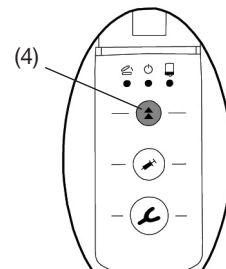
fig. 3

The unit is now ready for use.

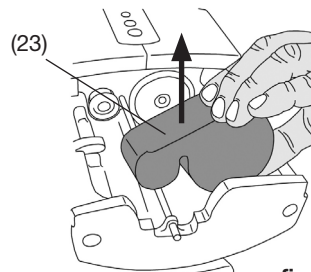
fig. 4

5. Operation

5.1 Cartridge Installation and Replacement

A cartridge can only be installed or replaced if both plungers (9, fig.1) are in their home position.

- Close the unit cover (8, fig. 1).
- Press the retract key (4).
- The plungers return to their home position and automatically stop.

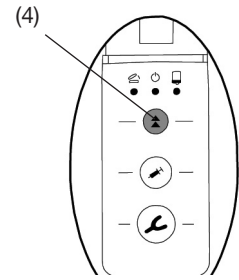
fig. 5

Just before the end, the retraction speed reduces.

Only open the unit cover once the plungers have reached their home position and have stopped.

- Open the unit cover (8, fig. 1).
- Place cartridge (20) in unit.
Please make sure that the cartridge is fitted into place under the catch (12).
- Select mixing program (15), cf. also section 5.1.1.
Information on the mixing program can be found on the cartridge in the form of an "asterisk" * or "diamond" ◆ symbol.

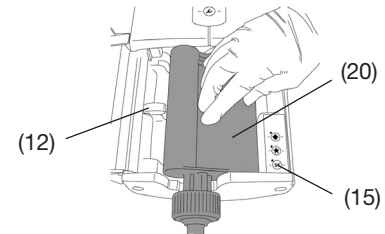
fig. 6

Any impression materials which do not have these symbols may only be processed using the "ST" program.

- Close the unit cover (8, fig. 1).
The cover will only close if the cartridge has been properly installed. The red "Unit cover open" LED (7, fig. 1) will remain on until the unit cover has been properly closed.
- Please refer to the instructions of the impression material manufacturer when preparing and leveling the cartridge systems.

Use only mixing nozzles recommended by the impression material manufacturer. The use of other mixing nozzles may cause malfunction.

5.1.1 Select mixing program

The mixing program is selected by pressing the program keys (15). The selection of a particular mixing program is confirmed by the green LED lamp next to the relevant program key.

The mixing programs marked with an asterisk * or diamond ◆ may only be used with special impression materials and cartridge/mixing nozzle systems which carry these symbols.

Any impression materials and cartridge/mixing nozzle systems not carrying these symbols must be processed with the "ST" mixing program.

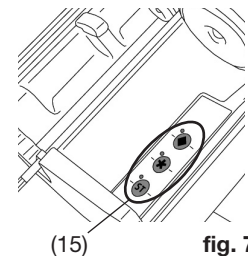
fig. 7

Please note that impression materials which are processed with the wrong mixing program will not meet the quality standards guaranteed by the respective manufacturer.

The following points apply when selecting the mixing program:

- Activate the mixing program by pressing the corresponding program key (15, fig 7).
The LED next to the program key goes on.

For mixing program the following applies:

- When the extrusion process has been completed, the mixing program selected remains in standby mode and is still enabled when the unit is switched off.
- When changing the cartridge, the plungers need to be moved to their home position and the mixing program is deactivated.
- When the plungers are in the home position, the unit cover is open and no mixing program is activated, the LED program key will flash.
- When the unit cover is closed, the LED will turn off. When opening the unit cover and no extrusion process has been started or no mixing program has been activated, the LED will start flashing again.
- If the operator presses one of the feed keys (impression tray (2, fig. 8) or dispenser (3, fig. 8)) without selecting a mixing program, the "ST" program will be activated automatically. The LED next to the "ST" key goes on.
- The operator can change from one mixing program to a different one at any time by pressing the relevant program key.

5.2 Extrusion

- Set up the unit as instructed in section 5.1.
- Press and hold down the advance key (2) or (3) (depending on the desired speed or function):
 - Impression tray (2), normal extrusion speed with activated mixing program or
 - Impression syringe (3), slow extrusion speed.
- The mixer shaft is activated in order to allow the hexagonal end (13, fig 1) to thread itself into the mixing nozzle.
- The plungers advance at high speed until they reach the impression material.
- Once the plungers' disc contact the impression material, the unit automatically switches to the selected extrusion speed.
- Fill the impression tray or impression syringe.
- Release the advance key as soon as the desired amount of material has been extruded.
- The plungers retract with a short relief stroke to prevent any additional impression material from being extruded.

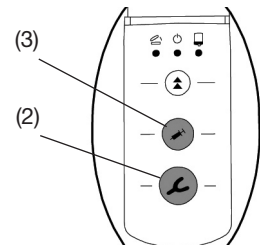
fig. 8

i *The plungers will only move if the unit cover is closed.*

i *The first 5 cm of impression material from a newly installed cartridge or from a previously used inserted cartridge should be discarded.*

Please note the impression material manufacturer's recommendations!

i *If the cartridge is emptied during an extrusion, the plungers automatically return to their home position so that a new cartridge can be quickly installed to start a new extrusion process.*

5.3 Fill Level Indicator and Remaining Amount

5.3.1 Fill Level Indicator

The fill level indicator (1) helps you to estimate the amount of impression material remaining in the cartridge. However, after installing a new cartridge or if the cartridge is very full, the red indicator will not be visible until the cartridge has started to be used.

5.3.2 Remaining Amount Detection

The Dynamix *speed* is equipped with an electronic fill level detection. Once there is only sufficient impression material remaining in a cartridge to fill approx. one casting spoon, the yellow "Residual level" LED (5) goes on. The indicator passes a residual level marking.

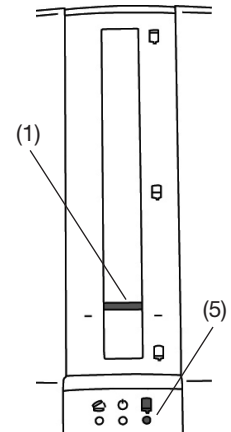
fig. 9

6. Cleaning / Maintenance

To clean the unit, wipe it down with a moist cloth.

! **Never use cleansers containing solvents.**

6.1 Cleaning the Interior

- Cartridge bed (14, fig. 1):
With the plungers (9, fig. 1) retracted, the cartridge bed is easily accessible for cleaning.
- Plungers' disc (9, fig. 1) and spindle covers (10, fig. 1):
With the unit cover closed and without a cartridge inserted the pistons may be moved into a position suitable for removing residues on the plungers' disc or the spindle covers.

! **NEVER clean the spindles (11, fig. 1) themselves. The grease applied to spindles is necessary for the unit's proper function.**

Tip:

Remnants of impression material are most easily removed with a dry cloth or paper towel.

6.2 Disinfection

! **Never use superheated steam for disinfection!**

Use only appropriate disinfectants for disinfection. Please pay attention to the manufacturer's instructions for the disinfectant used. If necessary, dry housing parts or exposed connectors after disinfecting.

6.3 Plungers' Disc Replacement

The plungers' disc (9, fig. 1) are subject to wear due to rubbing against the inner wall of the cartridge (20, fig. 1). Plungers' disc that are much worn may damage the foil bag in the supporting cartridge (20, fig. 1). Replacement plungers' disc are therefore available as a spare part.

The tool required for the replacement and a description of the procedure are enclosed with the spare part.

6.4 Unit Cover Replacement

The unit cover (8, fig. 1) is available as a spare part.

The tool required for the replacement and a description of the procedure are enclosed with the spare part.

6.5 Fuse Replacement


Before replacing the fuse, disconnect the unit from the main power supply by unplugging the power cord from the wall outlet.

Only replace fuses with new ones that have the same rating as those being replaced. Refer to the "Technical Specifications" section.

Never use fuses with a higher rating.

- Unplug the unit.
- Pull the fuse holder (17) out.
- Remove the defective fuse and insert a new one.
- Push the fuse holder back in, making sure it is completely seated.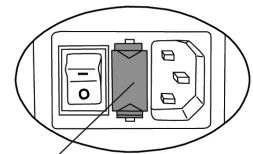

(17)

fig.10

6.6 Spare Parts

Please refer to the enclosed spare parts list for the order numbers of spare parts. Please state the item number and serial number of the unit in addition to the desired order number when ordering spare parts.

Serial number and date of manufacturing are shown on the type plate of the unit.

Accessories and Spare Parts

66033825 DYNAMIX WALL MOUNT (wall mount)

66034326 DYNAMIX UK POWER CORD 230 V (mains cable United Kingdom)

66034327 DYNAMIX CH POWER CORD 230 V (mains cable Switzerland)

66034592 DYNAMIX EU POWER CORD 230 V (mains cable EU)

66041021 DYNAMIX SPARE PART COVER NEW (NEW FORM) (spare part – unit cover new shape)

66032128 DYNAMIX SPARE PART PLUNGER DISC (spare part – plungers' disc)

66034329 DYNAMIX SPARE PART FUSE EU 230 V (spare part – fuse 230 V)

66047093 DYNAMIX SPARE PART BASE PLATE NEW (NEW COLOR) (spare part – unit plate new color)

66035030 DYNAMIX SPARE PART BASE FEET 4pcs (spare part – unit feet 4pcs)

66035051 DYNAMIX SPARE PART RUBBER STAMP (spare part – catch)

7. Scope of Supply

- 1 Dynamix *speed* Mixing unit, 230 V, 50 Hz.
- 1 Power cord.
- 1 Instructions for use.

8. Delivery Form

Item no.: 66045779 – Dynamix *speed* EU Standard, 230 V, 50 Hz.

9. Troubleshooting

Error	Cause	Corrective action
LEDs do not light LED (5, 6, 7, fig. 1), plungers will not move.	<ul style="list-style-type: none"> The unit is not switched on. 	<ul style="list-style-type: none"> Switch the unit on at the main power switch (17, fig. 1).
The unit does not switch on.	<ul style="list-style-type: none"> Power cord not properly connected to the unit. Fuse blown. No power at the wall outlet. Unit fault. 	<ul style="list-style-type: none"> Check that the unit plug is properly seated (fig. 2). Replace the fuse (refer to Sec. 6.5). Check that there is power at the wall outlet. Return the unit for repair.
Plungers will not move, "Unit cover" LED (7, fig. 1) is on or flashes.	<ul style="list-style-type: none"> Unit cover not properly closed. 	<ul style="list-style-type: none"> Close the unit cover (8, fig. 1) and check that the cartridges are properly seated. If required, seat cartridges correctly.
The plungers move slowly toward the cartridge.	<ul style="list-style-type: none"> The cover was opened before the plungers had stopped. A previous cartridge position has been saved. 	<ul style="list-style-type: none"> Press retract key (4, fig 1) and wait until the plungers have reached their home position and have stopped. Then repeat the process with the cartridge.
Cartridges cannot be removed / cannot be installed.	<ul style="list-style-type: none"> Plungers' disc have not fully retracted to their home position. 	<ul style="list-style-type: none"> Press the retract key (4, fig 1) and allow the plungers to retract until they stop automatically. They are now in their home position (refer to Sec. 5.1).
The hexagonal end (13, fig.1) fails to turn.	<ul style="list-style-type: none"> The mixer shaft has not threaded itself. Material has hardened in the mixing nozzle. The mixer motor's thermal protector has activated. 	<ul style="list-style-type: none"> Check that the cartridge is properly seated. Install a new nozzle. Allow the unit to cool down for approx. 30 minutes. Observe the maximum permissible operating time (2 minutes of operation, 5 minute pause).
Unit automatically reduces the extrusion speed.	<ul style="list-style-type: none"> Unit detects a highly viscous or too cold impression material (below 21°C [70°F]) and automatically reduces the extrusion speed and mixer rotary speed. Contamination (e.g. debris) in the cartridge holder. Insufficient spindle lubrication. 	<ul style="list-style-type: none"> This only occurs if the impression material is highly viscous or too cold. Refer to the manufacturer's processing instructions (processing temperature) or handle the impression material with a room temperature between 21 – 25°C [70 – 77°F]. Clean the inner surfaces of the cartridge holder and plungers' disc. Lubricate the spindles.
The mixing nozzle (21, fig.1) bursts.	<ul style="list-style-type: none"> Incorrect mixing nozzle installed. Mixing tip/ foil bag not compatible. 	<ul style="list-style-type: none"> Use the mixing nozzle specified by the manufacturer. Contact the manufacturer of the mixing tip / foil bag.
The tubular bag bursts or is damaged.	<ul style="list-style-type: none"> Plungers' disc are too much worn or have been damaged, causing the tubular bag to be pinched between the plungers' disc and the cartridge. The material has partially hardened in the cartridge. 	<ul style="list-style-type: none"> Replace the plungers' disc (refer to Sec. 6.3). Replace the cartridge (refer to Sec. 5.1).
Advance stops for no apparent reason and the unit performs a short relief stroke.	<ul style="list-style-type: none"> Material has hardened in the cartridge. 	<ul style="list-style-type: none"> Use new mixing nozzle.
Advance stops for no apparent reason and the pressure discs return to their home position.	<ul style="list-style-type: none"> The controller has detected an overload. Material has hardened in the cartridge. Insufficient spindle lubrication. 	<ul style="list-style-type: none"> Impression material is too cold. Refer to the manufacturer's processing instructions or handle the impression material with a room temperature between 21 – 25°C [70 – 77°F]. Use a new cartridge. Lubricate the spindles.
The mixing program LEDs are flashing while the unit cover is open.	<ul style="list-style-type: none"> No mixing program is selected. 	<ul style="list-style-type: none"> Select the correct mixing program for the impression material (see section 5.1.1).

10. Technical Specifications

Mains voltage:	230 V, 50 Hz 120 V, 60 Hz 100 V, 50/60 Hz
Protection class:	Class I (one)
Power consumption:	280 VA
Mains input fuse:	2 x 1.6 A (T) (230 V) 2 x 3.15 A (T) (100 V / 120 V)
Dimensions (height x width x length):	330 x 190 x 380 mm (13 x 7.5 x 15 inches)
Weight (empty):	7.0 kg
Noise level:	< 70 dB(A)

GB

- 10 -

11. Warranty

Provided the unit is properly used, Heraeus Kulzer **warrants** all the components of the *Dynamix speed* mixing unit **for a period of 2 years**.

Warranty claims may only be made upon presentation of the original sales receipt.

Components subject to natural wear (e.g. motor and gears) as well as consumables (e.g. fuses, plungers' disc and unit cover) are excluded from this warranty.

The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or in case of unusual influences or influences not in compliance with the instructions for use, as well as in case of modifications to the unit.

Warranty work on individual components shall not extend the original warranty of the overall unit.

We reserve the right to make technical changes.

12. Disposal Information

12.1 Disposing of Consumables

Empty cartridges and tubular bags as well as used mixing nozzles must be disposed of in accordance with the manufacturer's instructions.

12.2 Instrument Disposal

The instrument must be disposed of by a specialist facility.

12.2.1 Disposal Information for EU Nations

In order to maintain and protect the environment, to prevent environmental contamination and to improve the reutilization of raw materials (recycling), the European Commission has established guidelines according to which manufacturers must take back electrical and electronic devices in order to forward them to a regulated disposal or recycling system.

Within the European Union, devices identified with this symbol may therefore not be disposed of with unsorted municipal solid waste:

For more information regarding proper disposal please apply at your local authorities.

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