# Honigum-Mono

## Inhalt | Contents

Gebrauchsinformation – Deutsch	4
Instructions for use – English	14
Mode d'emploi – Français	24
Istruzioni per l'uso – Italiano	35
Instrucciones de empleo – Español	46
Instruções de utilização – Português	56
Gebruiksaanformatie – Nederlands	66
Brugsanvisning – Dansk	76
Bruksanvisning – Svenska	86
Instrukcja użycia – Polski	96
Информация по применению – Русский	107
Kullanım kılavuzu -Türkçe	118

## Produktbeschreibung

---

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Vinylpolysiloxane, Siliciumdioxid, Pigmente, Additive, Platinkatalysator

## Klassifikation

ISO 4823:2000, Type 2: Medium-bodied consistency

## Handelsformen

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Honigum-MixStar Mono	
1 Kartusche à 380 ml Paste, 10 MixStar-Tips	REF 909568
5 Kartuschen à 380 ml Paste, 50 MixStar-Tips	REF 909569

Honigum-Automix Mono

2 Kartuschen à 50 ml Paste, 4 Automix-Tips, 4 Intra-oral-Tips	REF 909830
8 Kartuschen à 50 ml Paste 20 Automix-Tips 20 Intraoral-Tips	REF 909834

4 Kartuschen à 50 ml Paste, 12 Automix-Tips, 12 Intra-oral-Tips	REF 999830
--	------------

Honigum-Handmix Mono	
2 Tuben à 135 ml Paste, 1 rutschfester Mischblock, 2 Tubenschlüssel	REF 909551

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Honigum-MixStar Mono	
1 Kartusche à 380 ml Paste, 10 MixStar-Tips	REF 999568

Honigum-Automix Mono	
----------------------	--

## Product Description

---

Honigum Mono is a monophasic impression material based on addition-curing silicones.

The material is available in three forms of application: In the ready-to-use pre-filled MixStar cartridge, which is suitable for use in the DMG MixStar-eMotion and other commercially available mixing machines, in the Automix safety cartridge and in a hand-mixed variant.

## Indications

---

- Crown and bridge impressions
- Inlay and onlay impressions
- Functional impressions
- Pressure point indication
- All types of pickup impressions, e.g. for implants and telescopes

## Basic Safety Instructions

---

- For dental use only!
- Keep out of reach of children!


## Interactions with other materials

---


- Rinse residues of retraction solutions and retraction pastes thoroughly with water, otherwise these can increase the setting time.
- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Never allow the impression material to come into contact with latex gloves. DMG recommends the use of PVC or PE based gloves.
- Do not mix Honigum-Mono with condensation curing silicone or polyether impression materials.

## Using the MixStar cartridge in the DMG MixStar-eMotion

- ▶ Follow the instructions for use for the “MixStar-eMotion”!

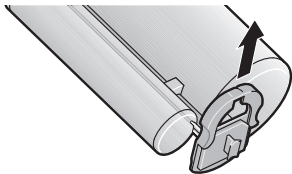
1. Slide the locking mechanism upwards in order to open the cartridge. Do not remove the locking mechanism from the cartridge.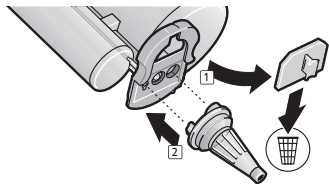


2. Remove the cap from the cartridge and discard. Attach a new mixing tip.

**Note:** In order to achieve an optimum mixing result, DMG recommends the use of mixing tips available from DMG. All mixing tips are available as refill packs.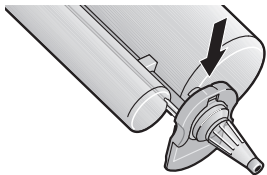

3. Secure the mixing tip by sliding the locking mechanism into the locked position.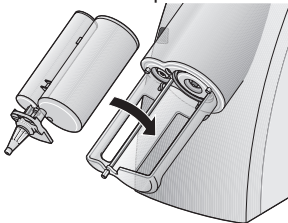

4. Insert the cartridge into the MixStar-eMotion. A brief acoustic signal confirms the correct insertion of the cartridge.

**Note:** Prepare fresh cartridges for use by extruding and discarding the first 3 cm of impression material.

**Note:** After the application, leave the used tip on the cartridge to cap unused material.

### Using the MixStar cartridge in mixing units from other manufacturers

---

- ▶ Follow the instructions of the mixing unit you are using!
- ▶ First insert the cartridge into the mixing unit without the mixing tip. Otherwise appropriate mixing of the material cannot be guaranteed!
- ▶ In this case, proceed as described under “Product Description” section on page 14.

## Changing the mixing tip

- ▶ Follow the instructions of the mixing unit you are using!


**Note:** Attach a new mixing tip before each mixing process.

1. Slide the locking mechanism upwards.
2. Remove the old mixing tip and any hardened material blocking the cartridge openings.
3. Place the mixing tip securely on the respective cartridge opening.

**Note:** In order to achieve an optimum mixing result, DMG recommends the use of mixing tips available from DMG. All mixing tips are available as refill packs.

4. Lock the mixing tip in place with the locking mechanism.

## Using the Automix cartridge

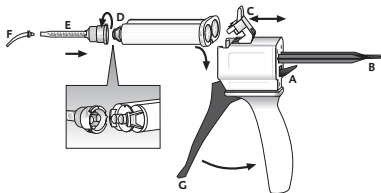### Insert cartridge

1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C] insert the cartridge and lock it using the plastic lever [C].

**Note:** Make sure that the notches on the cartridge line up with those on the Automix-Dispenser.

### Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

**Note:** In order to achieve an optimum mixing result, DMG recommends the use of mixing tips available from DMG. All mixing tips are available as refill packs.

**Note:** Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.
4. Attach an intraoral tip [F], if applicable.

### Applying the material

- ▶ When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [G] of the Automix-Dispenser and it is then applied directly.

**Note:** After the application, leave the used mixing tip on the cartridge to cap unused material!

### Remove cartridge

1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

### Handling the hand-mixing system

---

1. At room temperature, mix equal volumes or parts by weight of the base and catalyst paste on a mixing block using a spatula for 30 seconds until a fully homogenous, turquoise-colored mass forms.
2. Close the tube again immediately after use! In doing so, do not mix up the tube covers.

### Instructions for use

---

- ▶ The individual times can be found in the “Technical data” section on page 21.

- ▶ Honigum-Mono can be used in all common nonperforated impression trays.
- ▶ In order to improve the adhesion of the material to metal or plastic trays, a tray adhesive for A-silicon should be applied (e.g. Tray Adhesive, DMG).
- ▶ Best results are obtained with individual impression trays.

## Recommended use

---

### Monophase impression taking

1. Fill the prepared tray with Honigum-Mono. When doing so, hold the impression tray at a slight angle beneath the mixing tip.

**Please note:** To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the application process.

2. Fill a small amount of material into a commercially available elastomer syringe or use the 50-mL cartridge directly and syringe the preparation using an intraoral tip.

**Note:** The filling of the impression tray and syringing should be completed at the same time, because the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

3. Position the impression tray in the mouth within the working time, slowly and using very little force, and allow the impression to form.
4. Remove the impression from the mouth once the setting time has ended; refer to “Removal” section on page 20.

### Double-mix impressions

1. Fill the tray with Honigum-Mono. When doing so, hold the impression tray at a slight angle beneath the mixing tip.

**Note:** To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the application process.

2. Syringe the preparation with Honigum-Light. Follow the associated instructions for use!

**Note:** The filling of the impression tray and syringing should be completed at the same time, because the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

- ▶ Position the impression tray in the mouth within the working time, slowly and using very little force, and allow the impression to form.
- ▶ Remove the impression from the mouth once the setting time has ended; see “Removal” section on page 20.

## Removal

---

1. Remove the impression parallel to the longitudinal axis of the prepared teeth.
2. Then proceed as described under “Cleaning and disinfection of the final impression” section on page 20.

## Cleaning and disinfection of the final impression

---

- ▶ Only disinfect the final impression using dipping solutions.  
DMG recommends the use of a 2 % glutaraldehyde solution.
- ▶ Please ensure that the disinfectant used is indicated for use with addition curing silicone impression materials.
- ▶ Follow the instructions of the disinfectant solution’s manufacturer precisely!
- ▶ Leave the final impression to dry and store until casting; see “Storage of the impression” section on page 21.
- ▶ For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; [www.dgzmk.de](http://www.dgzmk.de)
- ▶ ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; [www.ada.org](http://www.ada.org)

## Casting and galvanization

- ▶ The impression should be cast 30 minutes at the earliest after removal from the mouth.

DMG recommends using special purpose hard plaster (ISO 6873; Type 3, e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the forms (ISO 6873; Type 4, e.g. Fujirock II, GC). The impression can be copper-plated or silver-plated.

## Storage of the impression

- ▶ Store impression at room temperature (15–25 °C/59–77 °F) in a dry place!
- ▶ Protect against exposure to direct sunlight!

The impression is stable and can be cast even after months without loss of accuracy.

## Technical data

Honigum-Mono			
Mixing time [min]	-	-	0:30
Working time <sup>1</sup> [min]	≤ 2:00	≤ 2:15	≤ 2:30
Intraoral setting time [min]	≥ 3:15	≥ 2:30	≥ 2:45
Dimensional change after 24 h [%]	≤ 0.4	≤ 0.4	≤ 0.4
Compression set [%]	≤ 0.70	≤ 0.70	≤ 0.70
Strain in compression [%]	≈ 3.0–5.5	≈ 3.0–5.5	≈ 3.0–5.5

<sup>1</sup>The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.

The working time describes the period from the start of mixing until the time by which the tray must be positioned in the patient's mouth.

## Disinfection of the Automix-Dispenser

---

The Automix-Dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix-Dispenser is also autoclavable.

## Storage and shelf life

---

- ▶ Store at room temperature (15–25 °C/ 59–77 °F) in a dry place.
- ▶ Do not store in the fridge or freezer!
- ▶ Protect against exposure to direct sunlight!
- ▶ After the application, leave the used mixing tip on the cartridge to cap unused material!
- ▶ Do not use after the expiration date!

## Composition

---

Vinyl polysiloxanes, silicium dioxide, pigments, additives, platinum catalyst

## Classification

---

ISO 4823:2000, type 2: Medium-bodied consistency

## Packaging

---

### Not available in the United States

Honigum-MixStar Mono	
1 cartridge @ 380 ml paste, 10 MixStar-Tips	REF 909568
5 cartridges @ 380 ml paste, 50 MixStar-Tips	REF 909569

Honigum-Automix Mono	
2 cartridges @ 50 ml paste, 4 Automix-Tips, 4 Intraoral-Tips	REF 909830
8 cartridges @ 50 ml paste, 20 Automix-Tips, 20 Intraoral-Tips	REF 909834

Honigum-Automix Mono	
4 cartridges @ 50 ml paste, 12 Automix-Tips, 12 Intraoral-Tips	REF 999830

Honigum-Handmix Mono	
2 tubes @ 135 ml paste, 1 nonskid mixing block, 2 tube keys	REF 909551

**Only available in the United States**

Honigum-MixStar Mono	
1 cartridge @ 380 ml paste, 10 MixStar-Tips	REF 999568

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**DMG** Chemisch-Pharmazeutische Fabrik GmbH