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# DuoTEMP

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04-2013

**Instructions for use (English)****Definition**

DuoTemp is a dual-cured, radiopaque, white-colored filling material for temporary filling of dental cavities. DuoTemp is a zinc oxide/zinc sulphate-based cement and is designed for short term temporary applications.

**Composition**

Zinc oxide  
Fillers  
Methacrylate resin  
Natrium fluoride

**Indications**

- Temporary fillings of class I and II cavities
- Temporary sealing in endodontics
- Temporary filling for inlay and onlay preparations

**Contra-indications**

Established allergy to one of the components of DuoTemp. Temporary filling of cavities, in which the cavity preparation has multiple surfaces and extends up to or under the gingiva (subgingival). DuoTemp should not remain in the oral cavity for more than one month. Due to the slight expansion of the DuoTemp, care must be taken in very deep cavities with thin enamel margins, as fractures may result.

**Side effects**

DuoTemp hardens by absorbing water. Dehydration in the cavity of vital teeth can cause short term pain. The cavity should therefore be kept moist using a water spray before applying DuoTemp. DuoTemp slightly expands during the curing process. This creates a very dense temporary filling with a good marginal seal, but care must be taken with very thin enamel margins as a fracture may result. Potential allergic reactions for methacrylate resins are possible among patients with sensitivity.

**Interactions**

DuoTemp bonds to other methacrylate-based products, e.g. with light cured cavity liner materials. Therefore, the cavity liner material should be isolated using a glycerine gel (e.g. Coltène Separator) to prevent it from being removed during removal. Interactions with other agents are unknown.

**Application of DuoTemp for temporary fillings and temporary endodontic sealing**

Moisten the cleaned cavity using water spray. Pick up pre-dosed DuoTemp from the pad using a spatula or an instrument, apply and wipe it into the cavity. Condense DuoTemp into the cavity using a condenser, and gently condense it to the cavity walls. It is recommended not to fill very deep cavities up to the base of the cavity to ensure that DuoTemp can be easily removed from the cavity.

**Optional**

For easier removal of the temporary filling, glycerine gel (e.g. Coltène Separator) can be applied before the application of DuoTemp in the cavity. Thus, the adhesion can be reduced.

**Application of DuoTemp for temporary fillings with an inlay or onlay preparation**

Apply glycerine gel (e.g. Coltène Separator) into the moistened cavity. Glycerin provides better moistening between the tooth and DuoTemp, and simplifies the removal from the temporary filling. The adhesion is also with glycerine ensured. Pick up predosed DuoTemp from the pad using a spatula or an instrument, apply and wipe it into the cavity. Condense DuoTemp into the cavity using a condenser, and gently condense it into the cavity walls.

**Removal of excess material**

Excess material should be removed before curing using a spatula or proper instrumentation.

## Curing

DuoTemp cures by water sorption (e.g. saliva action) within 2 h. The temporary filling can be immediately subjected to mastication pressure, if the surface is cured using all common curing light units (e.g. COLTOLUX® LED)

Recommended curing times:

- 40 s by > 800mW/cm<sup>2</sup>

- 60 s by < 800mW/cm<sup>2</sup>

DuoTemp is photosensitive, and should not be exposed to light too long, in particular to the operating light or sunlight. If possible, protect the pre-dosed DuoTemp material on the pad using a non light-transmitting covering (not blue colored).

## Handling

Remove occlusal contact points using a scaler.

The surface of the light-cured temporary filling surface can be carefully finished using proper rotary instrumentation.

## Removal of DuoTemp temporary fillings and endodontic sealing from the cavity

DuoTemp can be easily removed from the cavity in large pieces using a drill or a scaler. Rinse off any residue from the cavity using a water spray. Remaining residue can be easily removed using a brush.

## Removal of DuoTemp from inlay and onlay preparations

DuoTemp can be easily removed from the cavity in large pieces using a scaler. Rinse off any residue from the cavity using a water spray.

## Emergency measures

An acute danger of DuoTemp can be excluded due to the small quantity applied and the lack of toxicity. In the case of direct contact with the oral mucosa, rinsing with tap water is sufficient. In the case of contact with the eyes, rinse thoroughly with water (10 min), and consult an ophthalmologist showing these instructions for use.

## Note

Only supplied to dentists and dental laboratories or upon their instructions. Keep out of the reach of children! Should not be used after expiration date.

## Shelf life and marking

### Shelf life/storage

The syringe container furnished with a protective cover is sealed in aluminium-coated plastic bags (tropic-proof).

DuoTemp in pre-used syringes must be stored tightly sealed at temperatures not above 23 °C / 73 °F. Avoid exposure to direct sunlight or other heat sources.

DuoTemp fulfills the intended purpose until the expiration date, which is marked on the containers.

### Marking

The expiration date and **LOT** number are marked on the containers.

## Caution

Federal law restricts this device to sale by or on the order of a dentist.

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**Coltène/Whaledent AG**