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ALGINoplast® Instructions for use GB

regular set/fast set

Indications for use: ALGINoplast® is suitable for all impression techniques where alginates are indicated.

Description:
regular set: Yellow, dust free, alginate based powder.
fast set: Salmon-coloured, dustless powder on base of alginato.

Dosing accessories: spoon for powder and water beaker.

Procedure: Transfer the contents of the refill package into an air tight resealable container immediately after opening. Only one refill package at a time should be opened. Shake the container two or three times before use to loosen the powder.

Mixing ratio: 23 g powder : 50 ml water

Measuring: 1 (2) (3) (4) Lpepl ± 7,0 g (14,1 g) (21,2 g) (28,2 g) powder: 15,3 ml (30,6 ml) (46,0 ml) (61,2 ml) water.

ALGINoplast® powder from the container using the spoon provided and level the measure with a spatula. Place the powder into a mixing bowl and add the corresponding water. Three spoons of ALGINoplast® and the corresponding amount of water are generally required for larger complex impressions.

Met the powder and water to a homogeneous consistency using a mixing spatula. Refer to the manufacturer's instructions when using automatic mixers.

	regular set	fast set
Mixing time:	30 sec.	30 sec.
Working time: *)	90 sec.	60 sec.
Minimum setting time in the mouth:	90 sec.	60 sec.
Setting time: *)	180 sec.	120 sec.

Times with *) apply from the start of mixing at a temperature of 23 °C/73 °F for both powder and water with a water hardness of approx. 13°. These times are reduced by higher temperatures and lengthened by lower temperatures. Variations in water hardness may also result in deviations from these times (when using deionized water working and setting time are increased by 30 sec.)

Model fabrication: Maximum accuracy of the model is achieved by rinsing the impression thoroughly after removal from the mouth and pouring immediately, for example with MOLDANO®,

MOLDASTONE® or MOLDASYNT®. If longer storage is necessary, the impression should be kept in a moist atmosphere (closed box).

Disinfection: Impressions may be treated with standard disinfectant solutions. Please follow the respective manufacturer's instructions for disinfection and processing.

Product storage: ALGINoplast® should be stored in a closed air tight container in a dry area at maximum 25 °C/77 °F.

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