Dental Etching Agent

K-ETCHANT GEL

ENGLISH INSTRUCTIONS FOR USE

I. INTRODUCTION

K-ETCHANT GEL is an etching gel and consists of 40% phosphoric acid aqueous solution and colloidal silica.

II. INDICATIONS

Etching the enamel and dentin for adhesive restorations

III. PRECAUTIONS

- [Safety precautions]
 - If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching, or numbness, discontinue the use of the product and seek medical attention.
 - If the product comes in contact with human body tissues, take the following actions:
 - < If the product gets in the eye > Immediately rinse the eye with copious amounts of water and consult a physician.
 - < If the product comes in contact with the skin or the oral mucosa > The product may cause inflammation or erosion with its irritancy. Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and rinse with copious amounts of water.
 - Use caution to prevent the patient from accidentally swallowing the product.
 - To prevent cross infection, avoid sharing the same gel or brush tip among different patients.
 - Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity.

[Handling and manipulation precautions]

- The use of the product is restricted to licensed dentist.
- Do not use the product for purposes other than listed in INDICATIONS.
- When using this product, provide moisture control with a rubber dam if necessary.
- If the product adheres on clothes, wash off with water.
- The product may be hardened when it is not in use. In that case, the content returns to liquid as shaking the bottle with the cap.

[Storage precautions]

- The product must be used by the expiration date indicated on the package.
- The product must be stored at 2-25°C/ 36-77°F.
- The product must be stored in proper places where only dental practitioners can access it.

IV. COMPONENTS

Please see the outside of the package for contents and quantity.

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K-ETCHANT GEL

Principial ingredients:

- Phosphoric acid
- Water
- Colloidal Silica
- Dyol

V. CLINICAL PROCEDURE

- Dispense one drop of the product into a well of mixing dish and apply it to the entire cavity surface (enamel and dentin) with a small brush.
- After 30-60 seconds, wash thoroughly and dry with an air syringe.

[Caution]

Care should be taken to prevent saliva from coming in contact with the etched tooth surface. Any contaminated tooth surface with saliva etc. must be re-etched.

WARRANTY

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of the product to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

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Kuraray Noritake Dental Inc. 1621 Sakazu, Kurashiki, Okayama 710-0801, Japan