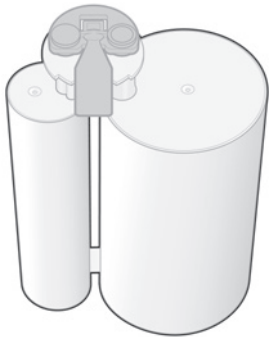
<b>D</b>	Gebrauchsanweisung .....	5
<b>GB</b>	Instructions for use .....	13
<b>F</b>	Mode d'emploi .....	21
<b>E</b>	Instrucciones de uso .....	29
<b>I</b>	Istruzioni per l'uso .....	37
<b>P</b>	Instruções de uso .....	45
<b>NL</b>	Gebruiksaanwijzing .....	53
<b>S</b>	Bruksanvisning .....	61
<b>DK</b>	Brugsanvisning .....	69
<b>N</b>	Bruksanvisning .....	77
<b>FIN</b>	Käyttöohje .....	85
<b>GR</b>	Οδηγίες χρήσης .....	93
<b>CZ</b>	Návod k použití .....	101
<b>HU</b>	Használati útmutató .....	109
<b>LV</b>	Lietošanas instrukcija .....	117
<b>LT</b>	Naudojimo instrukcijos .....	125
<b>PL</b>	Instrukcja użycia .....	133

Flexitime®

**Dynamix System, 380 ml**

Dynamix Cartridge, 380 ml

Dynamix Mixing Tip, yellow

Dynamix Fixation Ring

## Cartridge System, 50 ml

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## Instructions for use

**Elastomeric, silicone-based precision impression material, addition-cured.**

The Flexitime range is an optimally matched system and its different types of material can be combined for all impression techniques to suit specific requirements.

### Material

Flexitime Easy Putty  
Flexitime Dynamix Putty  
Flexitime Heavy Tray  
Flexitime Dynamix Heavy Tray  
Flexitime Monophase  
Flexitime Dynamix Monophase  
Flexitime Correct Flow

### Pack

Tub, 1:1, 2 x 300 ml  
Cartridge, 5:1, 380 ml  
Cartridge, 1:1, 50 ml  
Cartridge, 5:1, 380 ml  
Cartridge, 1:1, 50 ml  
Cartridge, 5:1, 380 ml  
Cartridge, 1:1, 50 ml

### Notes on use

Not to be used in conjunction with other silicone materials.

**Close tubs properly after use. Do not mix up caps/lids/measuring spoons.  
Dispense a small amount of material from a 50 ml cartridge before attaching a mixing tip.  
After use, leave the mixing tip on the cartridge to keep it closed. If switching to a new  
mixing tip, dispense a small amount of material before next used.**

With all addition-cured silicone impression materials changes in setting characteristics can occur if the material comes into contact with latex gloves, retraction solutions or freshly placed composite fillings.

Flexitime impression materials should be at room temperature (23°C) for processing. Lower temperatures, e.g. due to storage in the refrigerator, extend time in the mouth whilst higher temperatures shorten it.

Polyvinyl siloxanes are chemically resistant. Material which has not set can soil clothing.

### Contraindication

In very rare cases allergies may occur when using the product.

## Side effects

None known

Flexitime	EN ISO 4823	Product colour		Mixing ratio	Mixing time	Working time	Time in mouth	Shore A hardness	Deformation under pressure	Elastic recovery after deformation	Linear dimensional change (shrinkage)
		Base	Catalyst								
					s	min:s	min:s		%	%	%
<b>Flexitime Easy Putty</b>	Typ 0 Mouldable	Violet	Light grey	1:1	45	2:30	2:30	70	1.8	99.7	0.2
<b>Flexitime Dynamix Putty</b>	Typ 0 Mouldable	Violet	Light grey	5:1	Auto	2:30	2:30	68	1.8	99.5	0.2
<b>Flexitime Heavy Tray</b>	Typ 1 High viscosity	Violet	Light grey	1:1	Auto	2:30	2:30	72	1.6	99.0	0.2
<b>Flexitime Dynamix Heavy Tray</b>	Typ 1 High viscosity	Violet	Light grey	5:1	Auto	2:30	2:30	72	1.6	99.0	0.2
<b>Flexitime Monophase</b>	Typ 2 Medium viscosity	Purple	Light grey	1:1	Auto	2:30	2:30	58	2.6	98.8	0.2
<b>Flexitime Dynamix Monophase</b>	Typ 2 Medium viscosity	Purple	Light grey	5:1	Auto	2:30	2:30	58	2.6	98.8	0.2
<b>Flexitime Correct Flow</b>	Typ 3 Low viscosity	Light green	Light grey	1:1	Auto	2:30	2:30	42	4.6	99.6	0.1

The specifications of Heraeus products are based on extensive measurements in production, research and applications technology. We reserve the right to make slight product changes in the interests of technical progress.

**If there are substantial undercuts or highly divergent teeth in the impression area, appropriate blocking out must be performed if using Flexitime Heavy Tray or Easy Putty.**

**Model fabrication:**

The impression can be cast 30 min. after removal from the mouth – there are no other time constraints. The impression is stable in long-term storage.

**Recommended model material:**

All standard model materials can be used.

We recommend:

Class III plaster, MOLDANO®

Class IV plaster, MOLDASTONE®/MOLDASYNT®

**Electroplating:**

Impressions can be electroplated with copper and silver in conventional baths.

**Disinfection:**

Impressions can be treated with an aqueous sodium hypochlorite solution (5.25%, for up to 10 minutes\*) or with other disinfectant solutions suitable for silicones. When doing so, please observe the disinfection/processing instructions issued by the relevant manufacturers.

\*(DGZMK 3/93 V1.0)

### Adhesion:

We recommend using our Universal Silicone Adhesive.

### Storage conditions:

Do not use Flexitime impression materials if the expiry date has elapsed.

Do not store above 25°C.

### Dynamix System, 380 ml – step by step

Fig. 1  
Remove cap  
Insert cartridge  
into the mixer

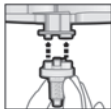
Fig. 2  
Attach Dynamix  
Mixing Tip

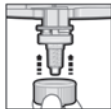
Fig. 3  
Position Dynamix  
fixation ring

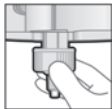
Fig. 4.1  
Attach Dynamix  
fixation ring  
Slide over recesses

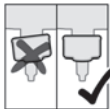
Fig. 4.2

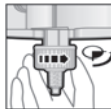
Fig. 5  
Lock by giving  
a ¼ turn  
clockwise

### Inserting the Dynamix cartridge:

For information on how to handle the mixer being used please refer to the detailed instructions issued by the relevant manufacturer.

- 
1. Hold the tab on the cap of the outlets and bend it upwards through 90° without applying any excessive force.
  2. Take the tab between your thumb and index finger whilst you hold it securely with your other hand and remove the cap from the cartridge completely. (Fig. 1)
  3. Place one of the enclosed Dynamix Mixing Tips over the outlets of the cartridge. (Fig. 2)
  4. If it proves difficult to attach the mixing tip, check to make sure that the central internal hexagon of the mixing tip is properly aligned with the hexagon of the drive spindle.
  5. When the mixing tip is in the correct position (Fig. 3), push the Dynamix fixation ring over the mixing tip until it stops and turn it clockwise (1/4 turn) until it is securely locked in place. (Figs. 4.1, 4.2 and 5)
  6. When impression material has been mixed, transfer it to a separate intraoral impression syringe, e.g. Impression Jet (Flexitime Dynamix Monophase only) or a suitable impression tray.
  7. When the material has been dispensed, retract the mixer plungers from the cartridge in order to avoid continuous pressure and thus prevent excess material from continuing to emerge from the mixing tip.
- 
- 

### **Notes about safe use of the Dynamix System**

Dynamix cartridges are heavy and can be damaged if they are dropped.

Damaged cartridges must no longer be used.

Only use Dynamix cartridges in mixers with a closed protective flap.

Before using for the first time, adjust the levels of the two material components.

Dispense approx. 2–3 cm material and discard this amount before starting to fill the tray.

Each time before you use the material check the outlet channels of the cartridge for blockages.

At the end of a working day or if the mixer is not going to be used for a lengthy period, remove the cartridge from the device and store it in a horizontal position with the outlets closed off by the mixing tip or cap.

**After use, leave the mixing tip on the cartridge to keep it closed.  
Immediately before the next use, remove the mixing tip and attach a new one.  
To do so, detach the Dynamix fixation ring and use it again for the new mixer attached.**

Only use Dynamix System components for dispensing and mixing the impression material in Dynamix cartridges.

Check the system components for physical damage on a regular basis and only use components which are intact.

Empty cartridges must be disposed of safely in accordance with EU Directives.

### **Warnings and safety instructions**

If material comes into contact with the eyes, rinse it out with water for several minutes. If symptoms persist, consult an eye specialist.

Do not swallow or consume. If symptoms occur after swallowing impression material, consult a doctor immediately. Bowel obstruction, for example, can occur in rare cases.

**For use by dentists only.**

Our applicational advice provided orally, in writing and by conducting tests is given according to the best of our knowledge but it is only to be regarded as non-committal information, also in respect of any third-party property rights, and does not relieve you of your duty to conduct your own tests on the products we supply with regard to their suitability for the intended procedures and purposes. Application, use and processing of the products are beyond our control so they are solely your responsibility. Naturally we give a warranty for the defect-free quality of our products in accordance with our Standard Terms and Conditions of Sale and Supply.

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Heraeus Kulzer GmbH
Heraeus Kulzer Inc.