

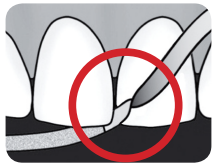
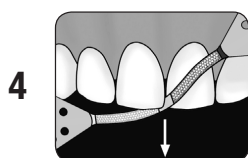
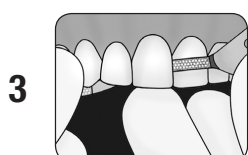
GripStrip Use Overview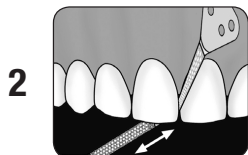

Twisting will cause strip to break.
Use wedge if too tight.

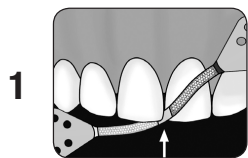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For professional use only

Product Description

- Metal strip, diamond-coated on one side only with two working sections of different grit sizes:
 - Coarse: 80 µm (blue) and 40 µm (red) – for substantial reduction
 - Medium: 40 µm (red) and 15 µm (yellow) – for moderate reduction
 - Medium Mini: 40 µm (red) and 15 µm (yellow) – for moderate reduction, reduced length for easier handling
 - Fine: 15 µm (yellow) and 8 µm (orange) – for fine reduction and polishing
- Uncoated middle section to easily slide through contacts
- Size/Dimensions:
 - 0.25mm height x 80mm length x 0.05mm thickness (Coarse, Medium, Fine)
 - 0.25mm height x 55mm length x 0.05mm thickness (Medium Mini)
- Perforated anatomical ends for better gripping
- Sterilizable and reusable

Indications of Use

	Coarse	Medium*	Fine
Shaping, contouring existing composite restorations	X		
Shaping composite restorations and crown margins in interproximal areas	X	X	
Finishing composite restorations and crown margins in interproximal areas		X	X
Removing excess material and overhangs from composite restorations and crown margins	X	X	
Final polishing of interproximal areas			X
Removing stains and discolorations in interproximal areas			X
Facilitating orthodontic movement of crowded teeth by opening interproximal spaces	X	X	X

*Same indications of use for Medium Mini

Instructions

- To move GripStrip through the contact, place the uncoated section of the strip at the contact (figure 1) and exert a slight horizontal movement.
NOTE: Do not twist, bend or pull strip too tightly or strip may break. For tight contacts, insert a wedge to create a slight separation between the teeth. To break a contact, use Centrix ProxDiscs® Proximal Separator Discs.
- Using a back and forward movement, first work the more abrasive side of the GripStrip (figure 2) followed by the less abrasive side to shape and smooth the tooth surfaces (figure 3) until desired result is reached. Fine GripStrips should only be used for polishing and stain removal.
NOTE: Skipping a grit size may reduce the quality of the final restoration.
- Remove the GripStrip by first positioning the uncoated segment at the contact (figure 4) and then slide out with a vertical movement.
NOTE: Do not remove the GripStrip through the contact area using the diamond-coated sides.
- The tabs at the end of each GripStrip enable easy grasping. When space is tight or more control is needed, grasp the GripStrip closer to the midline and use normally.

Maintenance and Sterilization

- GripStrip metal strips are supplied non-sterile. Prior to first use, and immediately after each use, the instruments must be disinfected or sterilized.
- Keep GripStrip metal strips separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- Use only cleaning/disinfecting solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer. **Rinse thoroughly with water after immersion into cleaning/disinfecting solution, otherwise discoloration or staining may occur.**
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and post-vacuum drying. Chemiclave sterilizers may also be used.

Caution and Warnings

- Twisting, rotating or excessive bending of GripStrip will cause the strip to break and can lead to detachment of the diamond coating.
- Pulling too forcefully in tight interproximal spaces can cause the strip to break.
- Leaving proximal surfaces roughened with 40 µm grit size or larger can lead to undesirable mechanical retention, formation of plaque and premature staining.
- Do not remove the GripStrip through the contact area using the diamond-coated sides.

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