(EN)

What is re:fresh?

Brilliant Smiles re:refresh is a simple teeth whitening treatment intended for in-office use. A gentle bleaching gel (6% hydrogen peroxide) is applied in a specially designed tray and is activated with blue plasma light.

Treatment duration

The teeth are normally treated for 2 x 20 minutes, but an additional 20 minutes may be added to the treatment session if needed (a total of 3 x 20 min).

Results

The result in all types of teeth whitening is individual and depends on the type and degree of discoloration. On average, re: fresh results in 5 shades whiter teeth in the treatment of 2 x 20 min and 6-7 shades in the treatment of 3 x 20 min.

Tip for best result!

Allow the patient to brush teeth with WHITENING^{boost} TOOTHPASTE from Brilliant Smile Sweden before commencing treatment. The toothpaste effectively removes coatings and surface stains, making the teeth whitening treatment more efficient.

Step by step instructions:

1. The patient cleans the teeth with Dental Cleanser, a cleaning cloth to put on the finger which removes saliva and coatings from the teeth. The patient should not close the mouth before inserting the tray, as saliva counteracts the bleaching process.
2. Apply a thin layer of gel on the foam strip of the tray and spread it into an even layer.
3. Apply vaseline to the patient's lips and then place the tray into the patient's mouth. The foam strip should be against the teeth and the lips should rest outside of the tray. Be sure to inform that the patient can and should swallow their saliva during the treatment.
4. Place the lamp in front of the mouth. The custom made tray has a magnifying glass in front, spreading the light in the mouth. The teeth are illuminated for 20 minutes at a time.
5. After 20 minutes, take out the tray, wipe dry with tissue and apply a new layer of gel. The patient can rinse the mouth during this time.
6. Dry the teeth with tissue, then again place the tray in the mouth and illuminate for 20 minutes. If necessary, this process is repeated once more.
7. When the light treatment is complete, remove the disposable tray. For sanitary reasons, the gel tip is removed and disposed after each patient.

Precautionary instructions

- ! The patient and the practitioner should wear safety goggles when light treatment is in progress.
- ! Do not use an excess amount of gel, as this may result in excess gel ending up on the gums causing irritation.
- ! Remove any excess gel on the lips or on the mucosal tissue in the mouth.

After the treatment

A sealing of the teeth with NanoSeal Total+ can be offered to patients who wish to be able to eat and drink anything right after treatment. The solution contains nanostructured fluoride which rebuilds the enamel immediately. NanoSeal Total+ is applied to dry and clean teeth after the teeth whitening treatment is completed.

- ! Without sealing, the following instructions are applicable:
72 hours after completion of the treatment, the patient should avoid acidic foods and drinks (cola, citrus drinks and fruit), as these can cause hypersensitivity. Also highly colored drinks (coffee, tea, red wine) and food (chocolate, tomatoes, curry powder, oranges, etc.) and tobacco should be avoided during the same time period, as this may compromise the results.

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