Provil® novo


Heraeus

DE Gebrauchsanweisung	3
GB Instructions for use	7
FR Mode d'emploi	11
ES Instrucciones de uso	15
IT Istruzioni per l'uso	19
PT Instruções de uso	23
NL Gebruiksaanwijzing	27
SE Bruksanvisning	31
DK Brugervejledning	35
NO Bruksanvisning	39
FI Käyttöohjeet	43
GR Οδηγίες χρήσης	47
CZ Návod k použití	51
HU Használati utasítás	55
LV Lietošanas instrukcija	59
LT Naudojimo instrukcija	63
PL Instrukcja obsługi	67
RU Инструкция по применению	71

Heraeus

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Cartridge Delivery System 2, 50 ml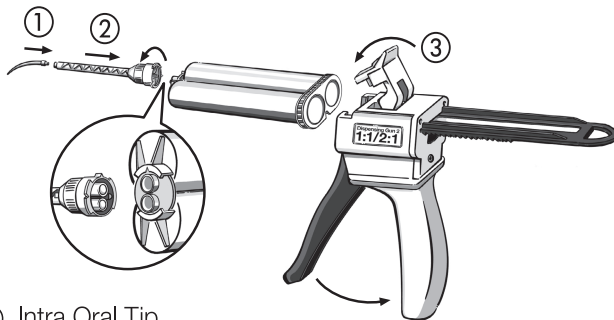

- ① Intra Oral Tip
- ② Mixing Tip
- ③ Dispensing Gun 2

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Stand: 2011-09

Elastomeric, silicone-based precision impression material, addition-cured

All products in the Provil novo assortment have a dosing ratio 1 : 1 (volume or weight).

Practical tips

Provil novo assortment is a variable consistency system that can be combined individually for all impression taking techniques. We do not recommend use in combination with any other silicone impression material. Visually inspect for damage before first use. Damaged products must not be used. Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases. No allergic reactions have been demonstrated; however the product may still generate an allergic reaction. For use by dental personnel only.

Tubes and Tubs must be carefully sealed after use. Do not inter-change lids, caps or spoons. When using cartridges extrude some material before positioning mixing tip. After use leave used mixing tip in situ as sealer on cartridge. Before next use remove sealed tip. Extrude some material and then fix new mixing tip.

As is common with all addition cured silicone impression materials inhibiting reactions with catalyst paste can occur when in contact with latex gloves, retraction cords or solutions and newly placed composite fillings. Therefore, please ensure that patient's mouth (i.e. preparation and gingival margin) is thoroughly washed and dried before impression taking commences.

	EN ISO 4823	Product colour	
		Base	Catalyst
Putty <i>fast set</i> Putty <i>regular set</i> Putty soft <i>fast set</i> Putty soft <i>regular set</i>	Type 0 putty	green green turquoise turquoise	light grey
Monophase <i>Tube</i> Monophase CD 2 <i>fast set</i> Monophase CD 2 <i>regular set</i>	Type 2 medium bodied	blue blue light blue	light grey
Medium <i>Tube</i> Medium CD 2 <i>fast set</i> Medium CD 2 <i>regular set</i>		yellow yellow light yellow	light grey
Light <i>Tube</i> Light CD 2 <i>fast set</i> Light CD 2 <i>regular set</i>	Type 3 light bodied	dark green dark green green	light grey

Mixing time [s]	Working time* [min:s]	Min. time in mouth [min:s]	Accessories	
			Mixing Tips	Intraoral Tips
30	1:30	2:30	–	–
30	2:00	3:30	–	–
30	1:30	2:30	–	–
30	2:00	3:30	–	–
30	2:30	3:30	–	–
–	1:30	2:30	green 66001989	white 65687514
–	2:00	3:30	green 66001989	white 65687514
30	2:30	3:30	–	–
–	1:30	2:30	yellow 66001988	yellow 66000782
–	2:00	3:30	yellow 66001988	yellow 66000782
30	2:30	3:30	–	–
–	1:30	2:30	yellow 66001988	yellow 66000782
–	2:00	3:30	yellow 66001988	yellow 66000782

*at room temperature of 23°C (73°F), 50% rel. humidity.
Higher temperatures shorten, lower temperatures prolong these times.

Side effects:	This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.
Model fabrication:	The impression can be cast 60 minutes after removal from the mouth – there are no other time constraints. The impression is stable in long-term storage.
Recommended Model material:	All standard model materials can be used. We recommend: Class III Gypsum, e. g. MOLDANO®; in the U.S.: Modern Materials®, Die Keen®; Class IV Gypsum, e. g. MOLDASTONE®/MOLDASYNT®
Electroplating:	Impressions can be electroplated with copper or silver in conventional baths.
Disinfection:	Impressions can be treated with an aqueous sodium hypochlorite solution (5.25%, for up to 10 minutes*) or with other disinfectant solutions suitable for silicones. When doing so, please observe the disinfection/processing instructions issued by the relevant manufacturers. *(DGZMK 3/93 V1.0)
Adhesion:	We recommend using our Universal Adhesive.
Storage conditions:	Do not store above 25°C (77°F).

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Heraeus Kulzer GmbH
Heraeus Kulzer, LLC