<div> <div></div> <div>Imprint™ 4</div> </div>
<div> <div></div> <div>VPS Impression Material</div> </div>
<div> <div></div> <div>Vinyl Polysiloxan Abformmaterial</div> </div>
<div> <div></div> <div>Matériau pour prise d’empreinte VPS</div> </div>
<div> <div></div> <div>Materiale per impronte a base di vinilpolisilossano</div> </div>
<div> <div></div> <div>Material de impresión de vinilpolisiloxano (VPS)</div> </div>
<div> <div></div> <div>Material de impressão VPS</div> </div>
<div> <div></div> <div>ENGLISH</div> </div>

Instructions for Use
Gebrauchsinformation
Mode d'emploi

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3M Deutschland GmbH Dental Products Carl-Schurz-Str. 1 41453 Neuss – Germany		<div> <div></div> <div>440007665/17/04</div> <div>14/06007665/04</div> </div>
<div> <div></div> <div>3M, ESPE, Imprint, Penta and Pentamix are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 2014, 3M. All rights reserved.</div> </div>		

ENGLISH

Product Description
Imprint™ 4 Impression material is a new generation of addition type silicones with improved hydrophilicity and peppermint flavor. Imprint 4 products provide very quick intraoral setting times. They are used for one-step and two-step impression techniques. The Imprint 4 portfolio consists of the following products:

Super Quick Setting Products:	ISO 4823	Consistency	Paste Color	Mixing Ratio (Vol.)	Mixing Device
Imprint™ 4 Penta™ Super Quick Heavy	Type 1	heavy-bodied	yellow	5:1	all Pentamix™ devices
Imprint™ 4 Super Quick Heavy**	Type 1	heavy-bodied	yellow	1:1	Garant Dispenser
Imprint™ 4 Super Quick Regular	Type 3	light-bodied	blue	1:1	Garant Dispenser
Imprint™ 4 Super Quick Light	Type 3	light-bodied	lavender	1:1	Garant Dispenser
Imprint™ 4 Super Quick Ultra-Light**	Type 3	light-bodied	orange	1:1	Garant Dispenser
Regular Setting Products:	ISO 4823	Consistency	Paste Color	Mixing Ratio (Vol.)	Mixing Device
Imprint™ 4 Penta™ Putty	Type 0	putty	pine green	5:1	All Pentamix devices after 1999.*
Imprint™ 4 Penta™ Heavy	Type 1	heavy-bodied	blue	5:1	all Pentamix devices
Imprint™ 4 Heavy**	Type 1	heavy-bodied	blue	1:1	Garant Dispenser
Imprint™ 4 Regular	Type 3	light-bodied	apple green	1:1	Garant Dispenser
Imprint™ 4 Light	Type 3	light-bodied	pink	1:1	Garant Dispenser
Imprint™ 4 Penta™ Monophase**	Type 2	medium-bodied	lavender	5:1	all Pentamix devices
Imprint™ 4 Monophase**	Type 2	medium-bodied	lavender	1:1	Garant Dispenser

* Pentamix devices before September 1999 extrude the paste to slowly.
** This product is not available in all countries.

These instructions for Use should be kept for the duration of product use. For all additionally mentioned products, please refer to the corresponding instructions for Use.

Indications

All types of impressions, e.g.:

- Crown, bridge, inlay, and only preparations
- Implants
- Orthodontic impressions

The super quick setting products are especially suited for single-unit and two-unit impressions.

Precautionary Measures

For Patients

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known mint oil allergies. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product.

For Dental Personnel

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water.

Use of protective gloves and a no-touch technique is recommended. Mint oil may penetrate commonly used gloves. If product contacts gloves, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed.

3M ESPE MSDSs can be obtained from www.mm.com or contact your local subsidiary.

Preparation

► Block out any undercuts or interdental spaces sufficiently to aid impression removal from the teeth after the impression material has set. Removal from the mouth may be difficult or could lead to the extraction of natural teeth or prostheses. Blocking-out may be necessary when using Imprint 4 Penta Putty or any other impression material with increased shore hardness.

Impression Tray:

All impression trays generally used for precision impressions are suitable. Rigid impression trays are recommended for two-step impression technique and when using Imprint 4 Penta Putty.

► For sufficient adhesion, apply VPS Tray Adhesive on the tray and allow it to dry completely. The ideal drying time is 5–15 minutes.

Penta Material:

- Place the Penta™ foil bags into the designated Penta™ cartridge.
- Insert the loaded cartridge into the Pentamix mixing unit.
- Attach a Penta™ Mixing Tip – Red.
- Turn the plungers of the mixing unit downward using the hand wheel until resistance occurs. Do not force further movement.
- When using new foil bags, start mixing and extrude a small amount until the paste is homogeneous in color. Discard this initial amount of paste.
- Please follow the section "Impression Taking" for tray filling procedure.

Garant Dispenser:

- Place the cartridge of the impression material to be used in a Garant dispenser.
- Prior to application, remove the cap and check that both cartridge openings are not clogged.
- When using new cartridges, extrude a small amount of paste without the mixing tip until base paste and catalyst are evenly extruded.
- To mix heavy-bodied or medium-bodied materials, attach a green Garant mixing tip to the cartridge.
- There are two options to mix and apply light-bodied or medium-bodied materials:
 - For light-bodied materials attach a yellow Garant mixing tip to the cartridge and a yellow Garant intraoral tip to the mixing tip. To enlarge the exit opening of the intraoral tip, cut it with a scalpel if required.

1.2 For medium- or bodied materials attach a green Garant mixing tip to the cartridge and a white Garant intraoral tip to the mixing tip. To enlarge the exit opening of the intraoral tip, cut it with a scalpel if required.

2. Use a 3M ESPE Intra-oral Syringe – Green. See next section.

► Please follow the section "Impression Taking" for the further procedure.

3M ESPE Intra-oral Syringe – Green

The Intra-oral Syringe – Green can be used for syringing preparations with VPS precision impression materials of ISO 4823 Type 3 and Type 2 consistencies, provided the VPS impression materials are intended for syringing.

- Have the Garant dispenser with the inserted cartridge ready for use (see Preparation Garant Dispenser).
- Have an Intra-oral Syringe – Green ready for use and remove the plungers. Leave the mixing tip of the Intra-oral Syringe in an angled position; do not fold open.
- Place the Intra-oral Syringe on the openings of the cartridge and hold firmly in place.
- Use the Garant dispenser to fill the Intra-oral Syringe with the required amount of impression material, but do not exceed the marking line (about 1 cm below the mixing tip joint).
- Remove the Intra-oral Syringe from the cartridge and close the cartridge, using the original cartridge cap.
 - When the cap is used more than once, make sure that the catalyst paste does not come into contact with the base paste to prevent paste carryover.
- Place the plunger in the Intra-oral Syringe, pushing the paste in the syringe barrel no further than to the joint of the mixing tip.
 - Any paste extruded on the sides of the mixing tip joint must be wiped off before the application in the mouth.
 - The content of an Intra-oral Syringe is sufficient for syringing about 2–4 preparations. It may be necessary to prepare a number of Intraoral Syringes in advance in the case of larger prosthetic work.
 - An Intraoral Syringe must only be used on one patient!
 - The filled Intra-oral Syringe can be stored for as long as 12 hours.
- Immediately before application, fold out the mixing tip of the Intra-oral Syringe at an angle of 180° (until resistance is felt).
 - The mixing tip must be folded out completely to prevent leakages at the mixing tip joint.
- Press out a small quantity of paste onto a mixing block until the base paste and catalyst flow out in an even, homogeneous mixture. Immediately apply the impression material around the preparation (see Impression Taking).

- Refilling an Intra-oral Syringe which has been emptied or partially emptied is prohibited.**

Retraction

- Retraction cords as well as solutions and pastes based on aluminum chloride, aluminum sulfate, or iron sulfate are suitable retraction agents.
- Recommendation: 3M ESPE Astrington Retraction Paste/Retraction Capsule. The paste containing aluminum chloride is directly applied from the retraction capsule into the sulcus. The paste can be used as an alternative to, or in combination with, retraction cords or other retraction methods, depending on the clinical situation and the practitioner's application technique.
- Areas from which impressions are to be taken should be kept dry.
- Prior to taking the impression, completely remove all residual retraction agents by rinsing and drying.

Times

	Working Time at 23 °C/74 °F	Intraoral Setting Time min:sec	Intraoral Setting Time min:sec
<div> <div></div> <div> <div> <div></div> <div> <div></div> <div></div> </div> </div> </div> </div>			
Super Quick Setting			
Imprint 4 Penta Super Quick Heavy	1:15	-	1:15
Imprint 4 Super Quick Heavy	1:15	-	1:15
Imprint 4 Super Quick Regular	1:15	0:35	1:15
Imprint 4 Super Quick Light	1:15	0:35	1:15
Imprint 4 Super Quick Ultra-Light	1:15	0:35	1:15
Regular Setting			
Imprint 4 Penta Putty	1:30	-	2:30
Imprint 4 Penta Heavy	2:00	-	2:00
Imprint 4 Heavy	2:00	-	2:00
Imprint 4 Regular	2:00	1:00	2:00
Imprint 4 Light	2:00	1:00	2:00
Imprint 4 Penta Monophase	2:00	1:00	2:00
Imprint 4 Monophase	2:00	1:00	2:00

Symbols on Product			
Super Quick Setting			
Imprint 4 Penta Super Quick Heavy	1:15	-	1:15
Imprint 4 Super Quick Heavy	1:15	-	1:15
Imprint 4 Super Quick Regular	1:15	0:35	1:15
Imprint 4 Super Quick Light	1:15	0:35	1:15
Imprint 4 Super Quick Ultra-Light	1:15	0:35	1:15
Regular Setting			
Imprint 4 Penta Putty	1:30	-	2:30
Imprint 4 Penta Heavy	2:00	-	2:00
Imprint 4 Heavy	2:00	-	2:00
Imprint 4 Regular	2:00	1:00	2:00
Imprint 4 Light	2:00	1:00	2:00
Imprint 4 Penta Monophase	2:00	1:00	2:00
Imprint 4 Monophase	2:00	1:00	2:00

* Start of mixing = Entry of paste into the mixing tip.

These are the processing times for products that have been stored at a temperature of 23 °C/74 °F and 20%–80 % relative humidity. These times are reduced due to higher temperatures of the product or prolonged if the temperature of the product is lower.

Impression Taking

When applying any kind of paste be sure:

- the impression tray is prepared with adhesive,
- the preparation is clean and dry,
- that the base paste and catalyst paste flow evenly into the mixing tip and extrude homogeneously in color and free of streaks (paste color: see "Product Description") and to discard any unevenly mixed paste.
- keep the mixing tip permanently immersed in the paste to avoid the formation of air bubbles,
- to apply the wash material in the sulcus from the bottom up,
- not to exceed the processing times of tray and syringe material to avoid incomplete and inaccurate impressions,
- not to extrude set material by force. This may damage the cartridge and mixing tip resulting in the formation of leaks.

One-step Impression Technique:

Recommended product combinations using Penta Tray Material

Imprint 4 Penta Super Quick Heavy Imprint 4 Super Quick Regular or Imprint 4 Super Quick Light

Imprint 4 Penta Putty Imprint 4 Regular or Imprint 4 Light

Imprint 4 Penta Monophase Imprint 4 Regular or Imprint 4 Light

Recommended product combinations using 50 ml cartridges

Imprint 4 Super Quick Heavy Imprint 4 Super Quick Regular or Imprint 4 Super Quick Light

Imprint 4 Heavy Imprint 4 Regular or Imprint 4 Light

Imprint 4 Monophase Imprint 4 Regular or Imprint 4 Light

- Load the impression tray with the heavy-bodied or Putty impression material.
- While the tray is being loaded, apply the wash material around the preparation using a wiggle and stir motion.
 - Depending on the number of units to be syringed, initiate application so that the tray loading and syringing around the preparation are completed simultaneously.
- Slowly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth and hold it in place without exerting pressure.
- Remove the impression from the mouth after the paste has set completely.

Monophase technique

- Initially fill the Penta™ Elastomer Syringe with Imprint 4 Penta Monophase or have Imprint 4 Monophase in the Garant Dispenser at hand.
- Load the impression tray with Imprint 4 Penta Monophase or Imprint 4 Monophase impression material.
- While the tray is being loaded, apply Imprint 4 Penta Monophase from the Penta Elastomer Syringe or Imprint 4 Monophase from the Garant Dispenser at hand, using a wiggle and stir motion.
 - Depending on the number of units to be syringed, initiate application so that the tray loading and syringing around the preparation are completed simultaneously.
- Slowly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth and hold it in place without exerting pressure.
- Remove the impression from the mouth after the paste has set completely.

Two-step Impression Technique:

Recommended product combinations

Imprint 4 Penta Putty Imprint 4 Super Quick Ultra-Light or Imprint 4 Light

- Load the impression tray with the heavy-bodied or Putty impression material.
- Slowly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth and hold it in place without exerting pressure.
- Remove the impression from the mouth after the paste has set.
- Excise undercuts and interdental seats, etc., and cut spillways if required.
- Clean and dry the impression thoroughly to provide for optimal cohesione of the two materials.
- Insert the wash impression material into the preliminary impression at the appropriate sites and/or apply around the preparations.
- Replace the preliminary impression in the mouth and hold it in place.
- Remove the impression from the mouth after the paste has set completely.

After Impression Taking

► Thoroughly examine and expose the sulcus of the prepared teeth and surrounding dentition. Remove any residual impression material that may have been left in the mouth.

Hygiene

- Place the impression in a standard disinfectant solution and follow the manufacturer's instructions for handling. Do not leave the impression submerged in the disinfection solution for more than the specified amount of time as otherwise the impression may be damaged.
- After disinfection, rinse the impression in any remaining disinfection solution with running water for approximately 15 seconds.

Model Preparation

- The impression may be poured at the earliest 30 minutes after removal from the mouth.
- In order to obtain a bubble-free model, rinse the impression with water shortly before pouring; remove excess water. A silicone surfactant can be used if required.
- Pour the impression with a standard dental die or model stone. The impressions can be silver- or copper-plated.

Cleaning

- Garant Dispenser: remove paste that has not set using a cloth soaked in alcohol. The dispenser handle and plunger can be autoclaved up to a temperature of 135 °C/275 °F. Disassemble the dispenser before autoclaving. Glutaraldehyde-based solutions can be used for disinfection.
- Paste that has not set may be removed with ethanol or by rinsing with water and soap.

Notes

- Completely remove the oxygen inhibition layer of composite materials, e.g., fillings or core buildups; it may impair or retard the setting of silicone impression material.
- Keep the filled mixing tip on top of the cartridge to serve as a closure until the next use. Removing and replacing the spent mixing tip may lead to carry-over of paste and ensuing formation of clogs.
- After filling a 3M ESPE Intra-oral Syringe replace the cartridge with the original cartridge cap.
- Vinyl polysiloxane impression materials must not be combined with condensation type silicones or polyether impression materials. Even small traces can affect the setting process.
- Disposable latex gloves impair the setting of silicone impression materials. Vinyl gloves are more suitable.
- Under no circumstances may impressions come into contact with solvent-containing fluids. This could result in expansion and inaccurate stone models.

Storage and Stability

Store the product at 15–25 °C/59–77 °F. Do not use after the expiration date. Store set impressions at temperatures below 30 °C/86 °F.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING AN IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy is 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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Aanbevolen productcombinaties uit de 50 ml cartridge					
Imprint 4 Super Quick Heavy	Imprint 4 Super Quick Regular	Imprint 4 Super Quick Light	Imprint 4 Super Quick Ultra-Light	Imprint 4 Regular	Imprint 4 Light
Imprint 4 Penta Putty	Imprint 4 Regular	Imprint 4 Regular of Imprint 4 Light	Imprint 4 Regular of Imprint 4 Light	Imprint 4 Regular of Imprint 4 Light	Imprint 4 Light
Aanbevolen productcombinaties uit de 50 ml cartridge					
Imprint 4 Super Quick Heavy	Imprint 4 Super Quick Regular	Imprint 4 Super Quick Light	Imprint 4 Super Quick Ultra-Light	Imprint 4 Regular	Imprint 4 Light
Imprint 4 Heavy	Imprint 4 Regular of Imprint 4 Light	Imprint 4 Regular of Imprint 4 Light	Imprint 4 Regular of Imprint 4 Light	Imprint 4 Regular of Imprint 4 Light	Imprint 4 Light
Imprint 4 Monophase	Imprint 4 Regular of Imprint 4 Light	Imprint 4 Regular of Imprint 4 Light	Imprint 4 Regular of Imprint 4 Light	Imprint 4 Regular of Imprint 4 Light	Imprint 4 Light

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Correcliechniek:

Aanbevolen productcombinaties					
Imprint 4 Penta Putty	Imprint 4 Super Quick Ultra-Light	Imprint 4 Light	Imprint 4 Regular	Imprint 4 Light	Imprint 4 Light
Aanbevolen productcombinaties					
Imprint 4 Penta Putty	Imprint 4 Super Quick Ultra-Light	Imprint 4 Light	Imprint 4 Regular	Imprint 4 Light	Imprint 4 Light

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